



BALLARD* ASSISTED CARE 24-HOUR ORAL CARE KIT

WITH SUCTION TOOLS

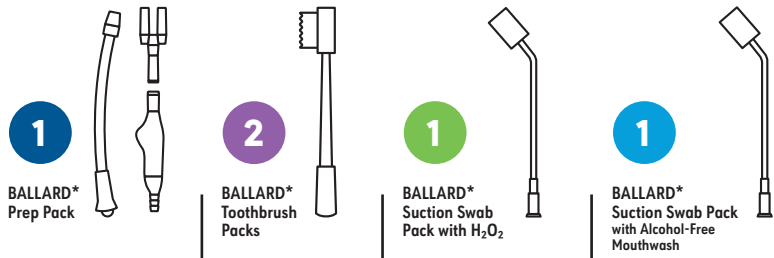

Instructions for Use: Oral care should be performed per institutional protocol.
****** For single patient use. Patients with altered levels of consciousness or who cannot comprehend commands may require use of a bite block. See instructions for use below.

PREP PACK ASSEMBLY

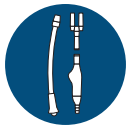

- Select BALLARD* Prep Pack from kit and open package.
- Connect stepped adapter of suction handle firmly to suction canister tubing. Attach Y-adapter firmly to suction port. Then firmly attach oral care suction tubing to one Y-adapter port and closed suction tubing to the other port. (Use the Y-adapter only if using a single suction canister for both closed suctioning and oral care.)
- Attach Yankauer firmly to suction handle.
- To store between uses, clip suction handle to top of suction tubing in upright position.

YANKAUER ASSEMBLY AND USE

- Attach Yankauer firmly to suction handle.
- Slide cleaning sleeve toward suction handle.
- To apply suction, slide switch to ON.
- Suction and clean oral cavity.
- After suctioning, advance cleaning sleeve toward tip to clean and dry the Yankauer.



- With suction still on, immerse Yankauer tip in sterile water or appropriate solution to clear Yankauer and suction tubing.
- Slide switch to OFF. (Switch should remain in OFF position when Yankauer is not in use.)
- To store, clip suction handle to top of suction tubing in upright position.
- **Note:** discard tubing, Y-adapter, suction handle, and Yankauer after using for 24 hours or according to protocol.

TOOTHBRUSH PACK with Antiplaque Solution SUCTION SWAB PACK with H₂O₂ SUCTION SWAB PACK with Alcohol-Free Mouthwash ASSEMBLY AND USE



Toothbrush or Suction Swab



- Select BALLARD* Toothbrush or Suction Swab Pack from kit and open package.
- Remove Yankauer from suction handle and set in component tray. Firmly attach tool.
- Peel back solution dispenser lid and moisten tool†
- To apply suction, slide switch to ON.
- Clean and suction oral cavity using appropriate tool according to hospital protocol.
- Slide switch to OFF.
- Remove tool from suction handle and discard.
- Reattach Yankauer firmly to handle and clip to suction tubing.

Applicator Swab and Mouth Moisturizer

- Remove applicator swab and coat with moisturizer.
- Apply moisturizer to lips and mouth.
- Discard after use.

† Suction Swab Pack with H₂O₂ provided for 24-hour period per Drug Facts.

******See graphics on the inner panel of the kit for a recommended order of use of components to support delivery of oral care by cleaning, refreshing, and stimulating the tissues of the mouth and throat, and subsequently suctioning excess liquids remaining after cleaning.

 Nonsterile	Disposable	 Not made with natural rubber latex
For use by a healthcare professional	Store at room temperature	REF 970600

Antiplaque Solution

Drug Facts	
Active Ingredient Cetylpyridinium Chloride 0.05%	Purpose Antigingivitis/Antiplaque
Uses <ul style="list-style-type: none">• Helps remove plaque that leads to gingivitis.	
Warnings <p>Stop use and ask a doctor if:</p> <ul style="list-style-type: none">• Gingivitis, bleeding, or redness persists for more than 2 weeks.• You have painful or swollen gums, pus from the gum line, loose teeth, or increasing spacing between the teeth. These may be signs or symptoms of periodontitis, a serious form of gum disease.	
Keep out of reach of children under 6 years of age.	
If more than used for rinsing is accidentally swallowed, get medical help or contact a Poison Control Center right away.	
Directions <ul style="list-style-type: none">• Adults and children 12 years of age and older: Use as directed by a dentist. Do not swallow the rinse.• Children 6 years to under 12 years of age: supervised use.• Children under 6 years of age: do not use.• Use only under health care practitioner's supervision.	
Other Information <ul style="list-style-type: none">• This rinse is not intended to replace brushing or flossing.• Store at room temperature.	
Inactive Ingredients Purified Water, Glycerin, Sorbitol, Flavor, Benzoic Acid, Sodium Benzoate, Sodium Saccharin, D&C Green 5	
Questions or Comments? 1-844-428-2667	

Mouth Moisturizer† Ingredients

Deionized Water, Glycerin, Soybean Oil, Carboxymethylcellulose (CMC), Polysorbate 80, Sorbitol, Vitamin E / Tocopherol Acetate, Spearmint Flavor, Sodium Benzoate, Xylitol, Citric Acid, Sodium Chloride

†Does not include petroleum.

Alcohol-Free Mouthwash Ingredients

Purified Water, Sorbitol, Glycerin, Flavor, Sodium Saccharin, FD&C Blue #1, FD&C Yellow #5, Cetylpyridinium Chloride, Benzoic Acid, Sodium Benzoate

Oral Debriding Agent

Drug Facts	
Active Ingredient Hydrogen Peroxide 1.5%	Purpose Oral Debriding Agent
Uses <ul style="list-style-type: none">• Aids in the removal of phlegm, mucus, or other secretions in the temporary relief of discomfort due to occasional sore throat and sore mouth.	
Warnings <p>Stop use and ask a doctor if:</p> <ul style="list-style-type: none">• Swelling, rash, or fever develop.• Severe or persistent sore throat or sore throat accompanied by high fever, headache, nausea, and vomiting occurs.• Do not use more than 2 days or administer to children under 3 years of age unless directed by a physician.	
Keep out of reach of children under 3 years of age.	
Directions <ul style="list-style-type: none">• Topical dosage for adults and children 3 years of age and older is a rinse used no more than 4 times daily. For children under 3 years of age, there is no recommended dosage except under the advice and supervision of a dentist or doctor.• Use only under health care practitioners supervision.	
Other Information <ul style="list-style-type: none">• Store at room temperature.	
Inactive Ingredients Purified Water, Glycerin, Flavor, Sodium Saccharin	
Questions or Comments? 1-844-428-2667	