Utopia® AV Full Face Mask

Instructions for Use – 06052 Rev H

Intended Use

The Utopia AV Full Face Mask is intended for use by adults (> 30 kg) as a patient interface for currently-marketed Continuous Positive Airway Pressure (CPAP) or Bi-Level positive-pressure ventilation devices for treatment of respiratory insufficiencies and obstructive sleep apnea in the hospital/institutional environment only. Dispose of the interface after single patient use.

Cautions


- In the USA, federal law restricts this mask to sale by, or on the order of, a physician. (RX Only)
- Storage Temperature: -20 to 60°C (-4 to 140°F), Operational Temperature: 5 to 40°C (-41 to 104°F), Humidity: 0–95%
- The deadspace associated with this mask is: SM – 299 cm³, MD – 336 cm³, LG – 392 cm³.

Warnings

- This mask **requires** a separate exhalation device.
- Do not use this mask in or near magnetic resonance imaging equipment, as it contains metal parts.
- A minimum pressure of 3.0 cm H₂O at the mask is required. The anti-asphyxia valve closes at operating pressures of 3.0 cm H₂O and higher.
- Some patients may have a reaction to the mask material. The cushion is made of silicone. Other mask materials include polycarbonate and polypropylene.
- Any unusual chest discomfort, shortness of breath, stomach distention, belching or severe headache during or immediately after use should be immediately reported to the physician.
- If oxygen is used with the CPAP or bi-level device, the oxygen flow must be turned off when the flow generator is not operating. When the CPAP or bi-level device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the flow generator enclosure. Oxygen accumulated in the CPAP or bi-level device enclosure will create a risk of fire. This warning applies to most types of CPAP or bi-level flow generators.
- Do not smoke while oxygen is in use.
- Note: At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate. This warning applies to most types of CPAP systems.
- This mask is intended for single-patient use only and thus use by multiple patients is to be avoided due to the risk of transmitting infection. Do not attempt to sterilize or use disinfectants on this mask.

Contraindications

This mask may not be suitable for persons with the following conditions: impaired cardiac sphincter function, excessive reflux, impaired cough reflex, and hiatal hernia. It should not be used if patient is uncooperative, obtunded, unresponsive, or unable to remove the mask.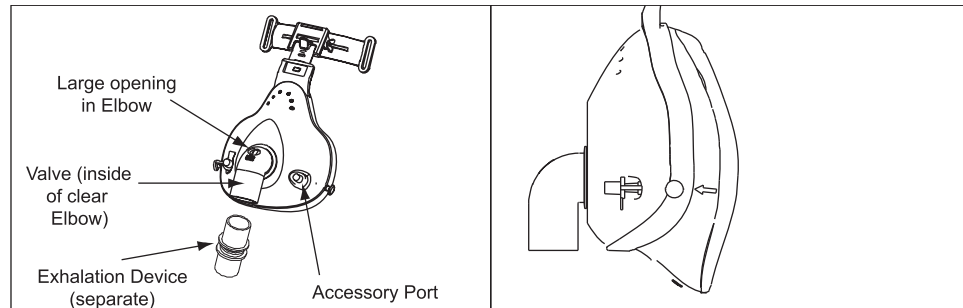

Instructions for Use

- Prior to use, clean the mask as described in the cleaning instructions.
- Clean and thoroughly dry the patient's face before putting on the mask.
- Inspect the anti-asphyxia valve located within the clear elbow. With the CPAP or bi-level system turned off, verify that the valve is positioned so that room air can flow through the large opening. Turn on the system; the valve should close the large opening, and system air should flow into the mask.


Warning: Do not use the mask if the anti-asphyxia valve is missing, damaged, or not working properly because it allows the patient to breathe room air if the CPAP/bi-level device failed. If the valve does not close or does not function properly, replace mask. Do not block the large opening at the valve.

- Attach the headgear to the forehead support by inserting the straps with the Velcro tabs through the slots on the support. The straps should be led with the "hook" side facing outward so they can be folded back and secured against the "loop" surface of the strap.
- Place the mask over mouth and position the cushion flap under the chin. Pull the headgear back over the top of the head. Adjust the straps on the forehead support until it fits snug against the face.

Warning: A mask should not be used unless the CPAP or bi-level system is turned on and operating properly. This warning applies to most CPAP devices. At low flows, the patient may breathe in some of the air that he has exhaled.


Warning: Do not use the mask if the patient is vomiting or feeling nauseous. Failure to arouse and remove the mask after vomiting could result in aspiration of vomitus. To minimize the risk of vomiting during sleep, the patient should avoid eating or drinking for three hours prior to sleep. This mask is not recommended if the patient is taking a prescription drug that may cause vomiting.

- Attach the quick-release buckles to the pegs on the mask and pull the buckle back (towards the face) until an audible "click" is heard.
- Attach the circuit tube from the CPAP or bi-level ventilation system to the elbow. **NOTE: If the circuit tube does not have an exhalation device, a separate exhalation device must be placed between the circuit and the elbow.**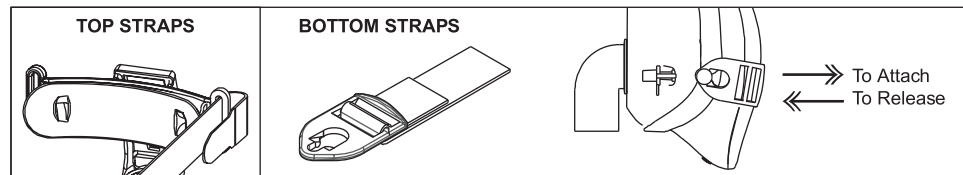

Adjusting and Removing the Mask

- To adjust the mask, gently pull each headgear strap until the mask sits squarely on the patient's face. **NOTE:** The seal on the mask cushion is designed to inflate towards the face, so generally only light tension on the straps is required. Over-tightening may cause leaks.
- The height of the forehead brace can be adjusted by pressing on the thumb lever, and moving the brace to a different slot in the support link.
- To adjust cushion pressure on the nose bridge, bend the support link where it exits the mask shell. **NOTE:** To reduce pressure on the nose bridge, bend the support link forward so that the forehead brace is closer to the face.
- To remove the mask, remove the headgear straps by pressing the tab on the quick release buckles forward, away from the face. Keeping the straps on the forehead support in place, lift the mask over patient's head.

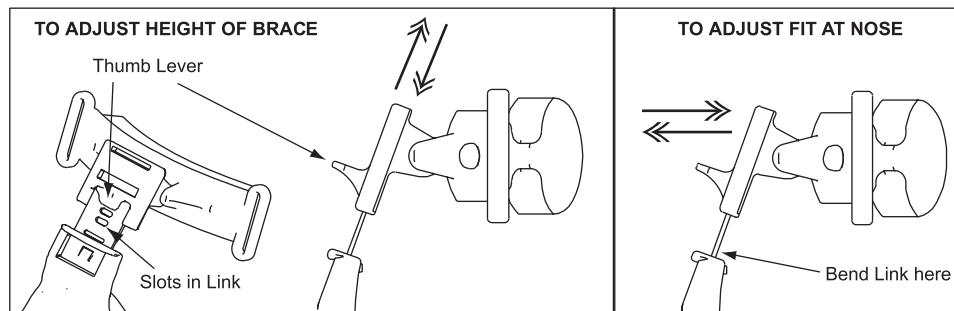Cleaning Instructions

After Each Use:

- Remove the headgear from the mask as the headgear will be washed separately. The headgear should be handwashed following the remaining instructions at least every 7 days. Do not machine wash.
- Handwash all components in warm, soapy water. **NOTE:** Only use pure soap and do not use alcohol, petroleum products, or strong cleaners as these may cause damage and reduce the life of the product.
- Rinse thoroughly and shake excess water from interior surfaces of components. Allow all components to completely air-dry, away from direct sunlight.
- Carefully inspect each component for any wear or damage. Discontinue use if you notice any cracks in the plastic, any tears or hardening of the silicone, or significant fraying or loss of elasticity of the headgear, and contact your healthcare provider for a replacement. Discoloration is normal after several washings and should not be considered sign of wear.

Manufactured for:

30 Spur Dr., El Paso – TX 79906 – USA
Tel: (800) 421-0024 – www.salterlabs.com

Covered by Patents: U.S. 8789532. Other patents pending.
Utopia® is a U.S. registered trademark.

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