LOT

EN: ENGLISH

## Sylent™ NE Nasal Mask

Instructions for Use – 09004 Rev F

### Intended Use

The Sylent NE Nasal Mask is intended for use by adults (> 30 kg) as a patient interface for CPAP or Bi-Level positive-pressure ventilation devices for treatment of respiratory insufficiencies and obstructive sleep apnea in the hospital/institutional environment only. Dispose of the interface after single patient use.

The deadspace associated with this device is less than 139 mL.

Storage Temperature: -20° to 60°C (-4° to 140°F)

Humidity: 0–95%

### Warnings

- This mask **requires** a separate exhalation device.
- In the USA, federal law restricts this mask to sale by, or on the order of, a physician. (Rx ONLY)
- The mask should be used only with the ventilation system recommended by your physician or respiratory therapist. A mask should not be used unless the ventilation system is turned on and operating properly. The exhalation port(s) of the separate exhalation device should never be blocked.
- At low CPAP pressures, the flow through the exhalation ports may be inadequate to clear all exhaled gas from the tubing. Some rebreathing may occur.
- If you experience any reaction or notice any redness or irritation of your skin, discontinue use and contact your physician. The mask consists of silicone, polycarbonate, polypropylene, nylon, polyester, and neoprene.
- This mask is intended for single-patient use only and thus use by multiple patients is to be avoided due to the risk of transmitting infection. Do not attempt to sterilize or use disinfectants on this mask.
- If oxygen is used with the ventilation system, the oxygen flow must be turned off when the ventilation system is not operating. *Explanation of the warning:* When the ventilation system is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the ventilation system enclosure. Oxygen accumulated in the ventilation system enclosure will create a risk of fire. This warning applies to most types of ventilation systems. Do not smoke while oxygen is in use. Note: At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate. This warning applies to most types of ventilation systems.

### Assembling the Mask

- Prior to use, clean the mask as described in the cleaning instructions.
- Before attaching the cushion, notice that the cushion and the shell both have an arrow marking on the bottom surfaces. Line up the arrow markings and then press the cushion completely onto the shell.
- Press the forehead support into the shell until the small button on the forehead support goes into the lowest slot on the shell.
- Attach the headgear to the forehead support by inserting the two loose straps (with the hooks facing outward) into the slots on the forehead support and then folding the ends back to secure the hooks to the "loop" surface of each strap.

### Fitting the Mask

- Clean and thoroughly dry your face before putting on the mask.
- Turn on your ventilation system and make sure it is functioning properly.
- Attach the circuit tube of the ventilation system to the elbow. *NOTE: If the circuit tube does not have an exhalation device, a separate exhalation device must be placed between the circuit and the elbow.*
- Place the mask lightly over the nose and pull the headgear back over the top of the head.
- Attach the two lower buckles of the headgear to the cylindrical portion of the posts on the mask and pull the buckles back (towards the face) until an audible "click" is heard.
- Unhook the hook-and-loop closures of each of the headgear straps in pairs (top pair and bottom pair), gently pull until the mask sits squarely on your face, and then reconnect the hooks. The mask should fit snug enough to prevent large leaks yet still be comfortable. In general, a looser fit is better. Over-tightening may actually make leaks worse and create discomfort.
- The location where the forehead support contacts the face can be adjusted vertically in order to further optimize comfort and seal. If desired, the forehead support may be pulled up where it joins the shell in order to place its small button into the middle or top slot on the shell.
- Breathe normally, and continue making minor adjustments until you have a comfortable fit with minimal leaks. If you are unable to achieve a comfortable fit or to resolve any leaks around the nose, you may need an alternative cushion size.
- To remove the mask, you do not need to unhook each strap. Just unsnap the two lower buckles of the headgear from the mask, and then lift the mask off over the head.

### Cleaning Instructions

#### After Each Use:

- Remove the headgear from the mask as the headgear will be washed separately. The headgear should be handwashed following the remaining instructions at least every 7 days. Do not machine wash.
- Remove the cushion and the forehead support from the shell.
- Handwash all components in warm, soapy water. NOTE: Only use pure soap and do not use alcohol, petroleum products, or strong cleaners as these may cause damage and reduce the life of the product.
- Rinse thoroughly and shake excess water from interior surfaces of components. Allow all components to completely air-dry, away from direct sunlight.
- Carefully inspect each component for any wear or damage. Discontinue use if you notice any cracks in the plastic, any tears or hardening of the silicone, or significant fraying or loss of elasticity of the headgear, and contact your healthcare provider for a replacement. Discoloration is normal after several washings and should not be considered sign of wear.

Manufactured for:

Salter Labs 30 Spur Dr., El Paso, TX 79906  
USA Tel: (800)-421-0024 [www.salterlabs.com](http://www.salterlabs.com)

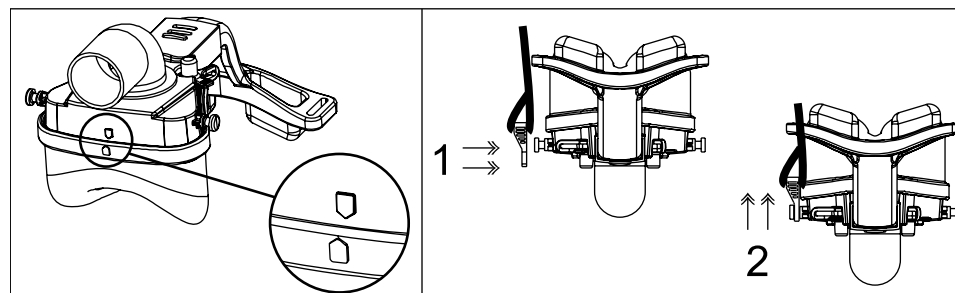
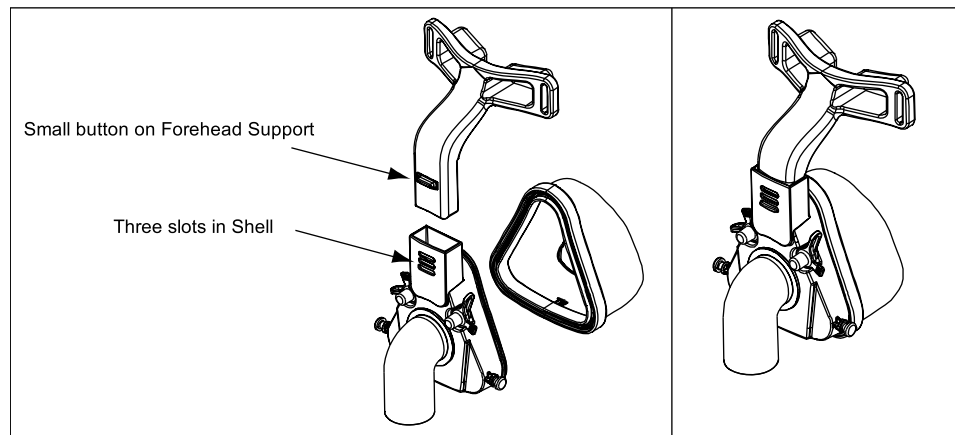
Manufactured by:

GaleMed (Xiamen) Co., Ltd.  
Xiamen Area of China (Fujian) Pilot Free Trade Zone, 39, Section 3,  
Haijing East Road 361026 Fujian Province, China  
[www.galemed.com](http://www.galemed.com) Tel: 86-592-6880588  
Made in China

Sylent is a trademark of Salter Labs, Inc. \* Covered by Patents D624642 and D626215. Other Patents pending\*  
Made in China

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Sylent è un marchio registrato di Salter Labs, Inc. \* US D624642, D626215; IN ATTESA DI BREVETTAZIONE\*