1101 Lucas • Suite 200 • St. Louis, MO 63101-1159 USA
Phone: 314-781-5700 • Fax: 314-644-4262
email: airQ@cookgas.com
www.cookgas.com

Product of Malaysia

air-Q Blocker® MASKED LARYNGEAL AIRWAY

Instructions for Use

The air-Q Blocker Airway is an enhanced version of the standard air-Q Blocker masked laryngeal airway. As such, it is indicated as a primary airway device when an oral endotracheal tube (OETT) is not necessary or as an aid to intubation in difficult airway situations. It is especially suited for applications when a bite block and / or access to the posterior pharynx and upper esophagus is desired.

This product is to be used by trained personnel only

Thank you for purchasing the air-Q Blocker Masked Laryngeal Airway by Cookgas® LLC. Due to its patented design, the air-Q Blocker is user-friendly. Placement is easy, air movement is outstanding, and intubation using standard oral endotracheal tubes (OETT), sizes 8.5mm - 5.5mm is straightforward and reliable. air-Q Blocker removal following intubation is quickly accomplished using the patented air-Q Blocker Removal Stylet, also by Cookgas, LLC.

Welcome to the Next Generation of Airway Management! Say Goodbye to the Difficult Airway, and Hello to the air-Q Blocker.

**The Only Airway You'll Want,
The Only One You'll Need!**

Available in Single Use

Recommendations:

Size	IBW	Max. OETT	Mouth Opening ¹	Volume ³	Inf. Vol. ⁴	
4.5	70-100 kg	8.5mm	25 mm	20 cm	25 ml	4-5 ml
3.5	50-70 kg	7.5mm	23 mm	18 cm	18 ml	3-4 ml
2.5	30-50 kg	6.5mm	20 mm	16 cm	12 ml	2-3 ml

- Minimum mouth opening for insertion.
- Distance from the external edge of the airway tube to the internal ventilatory opening.
- Internal volume from the external edge of the connector to the internal ventilatory opening.
- Recommended cuff inflation volume following insertion with inflation valve open to air.

air-Q Blocker Placement Procedure

The procedure below is intended as a guide. Many techniques can be successfully used to properly place the air-Q Blocker into the pharynx.

- Open the inflation valve to air (either leave the red tag on or insert an empty syringe barrel into the inflation valve). Lubricate the external surface including the mask cavity ridges.
- Open the patient's mouth and elevate the tongue. Elevating the tongue lifts the epiglottis off the posterior pharyngeal wall and allows the air-Q Blocker easy passage into the pharynx. A mandibular lift is especially recommended. A tongue blade placed at the base of the tongue also works well for this purpose.
- Place the front portion of the air-Q Blocker mask between the base of the tongue and the soft palate at a slight forward angle, if possible.
- Pass the air-Q Blocker into position within the pharynx by gently applying inward and downward pressure, using the curvature of the air-Q Blocker mask and airway tube as a guide. Simply rotate the air-Q Blocker forward and inward. Minimal manipulation may be necessary to turn the corner into the upper pharynx. Continue to advance until fixed resistance to forward movement is felt. Correct placement is determined by this resistance to further advancement. Some users place the back of the left index finger behind the mask, flexing the finger forward to help guide the mask around the corner into the pharynx. Once the mask has negotiated the turn, the left hand is then used to do a mandibular lift while exerting downward and inward pressure on the air-Q Blocker with the right hand during final advancement into the pharynx. This technique seems to be easy to learn and is particularly successful.
- Tape the air-Q Blocker in place and inflate the air-Q Blocker cuff according to Recommendations Table. Do not overinflate. Cuff pressure <60 cm H₂O, ideal 20-30 cm.
- Check the air-Q Blocker connector to ensure it is fully engaged within the airway tube, and attach the connector to the appropriate breathing device. Check for adequate ventilation.
- Place a bite block between the patient's teeth. Keep the bite block in place until the air-Q Blocker is removed.

air-Q Blocker Intubation Procedure

The air-Q Blocker by Cookgas® LLC is intended not only to be an outstanding airway for general use, but also to be a simple and reliable tool for intubation of the trachea with OETT's. Due to its patented design, standard OETT's (sizes 8.5mm - 6.5mm) can be easily passed through the air-Q Blocker and into the trachea. Further, the air-Q Blocker can be easily removed following intubation with the aid of the patented air-Q Blocker Removal Stylet, also by Cookgas®, LLC. The following procedure for intubation is only intended as a guide. Many techniques can be successfully used for tracheal intubation using the air-Q Blocker.

- Prior to intubation, the laryngeal musculature and vocal cords must be relaxed, either by an aerosolized local anesthetic or with the aid of a muscle relaxant.
- Pre-oxygenate.
- Prepare the appropriately-sized OETT by completely deflating the OETT cuff and lubricate well. It is important to deflate the OETT cuff completely to allow the OETT to slide easily within the air-Q Blocker.
- Disconnect the air-Q Blocker from the airway device and remove the air-Q Blocker connector. This can be easily done by squeezing the air-Q Blocker tube between the index finger and thumb just distal to the connector with one hand, then rocking the air-Q Blocker connector back and forth while pulling the connector outward away from the airway tube with the other.

- Insert the previously deflated and lubricated OETT through the air-Q Blocker to a depth of approximately 8 - 20 cm, depending on the air-Q Blocker size. This will place the distal tip of the OETT at or just proximal to the opening of the air-Q Blocker airway tube within the mask cavity. It is very important to lubricate the OETT and the air-Q Blocker airway tube completely to ensure easy passage of the OETT through the air-Q Blocker.
- The following suggestions for advancement of the OETT are intended as a guide. Many techniques can be successfully used to further advance the OETT into the trachea.

CAUTION: Always check for adequate ventilation and oxygenation following placement of the OETT.

- Fiber Optic Technique:** Using a Fiber Optic Endoscope, pass the scope through the OETT and into the trachea under direct visualization. Stabilize the Fiber Optic Endoscope and pass the OETT through the laryngeal inlet and into the proximal trachea, using the scope as a guide. Check the position of the OETT with direct visualization of the tracheal carina. Remove the Fiber Optic Endoscope. Add a small amount of air to the OETT cuff, then replace the OETT connector if necessary. Check for adequate ventilation. (If epiglottic intrusion or down-folding is seen during visualization, the air-Q Blocker usually need not be completely removed. Perform an external mandibular lift to elevate the epiglottis and pass the endoscope beneath the epiglottis and into the trachea followed by the OETT.
- Stylet Technique:** Using an appropriate coude tipped intubating stylet or a lighted stylet, pass the intubation stylet through the OETT within the air-Q Blocker, through the laryngeal inlet and into the trachea. Pass coude-tipped stylets with tip pointing upward (anterior). By gently placing the fingers of the left hand over the cricoid area of the patient's throat, the stylet can usually be felt as a scraping or rubbing sensation as it passes through the cricoid ring. If properly positioned, the lighted stylet will also produce a bright yellow/red illumination over the cricoid area. Once the stylet passes into the trachea, simply advance the OETT over the stylet, through the laryngeal inlet and into the trachea, using the intubation stylet as a guide. Add a small amount of air to the OETT cuff, replace the OETT connector and check for adequate ventilation. **NOTE:** If the OETT fails to advance over the stylet into the trachea, it is usually helpful to rotate the OETT counter clockwise while passing the OETT. If this fails, try again with a smaller size OETT.

air-Q Blocker Removal Procedure

Removing the air-Q Blocker following OETT intubation is easily accomplished with the aid of the air-Q Blocker Removal Stylet by Cookgas®, LLC. The air-Q Blocker Removal Stylet consists of an adapter connected to a rod. The adapter is tapered from bottom to top, and has horizontal ridges and vertical grooves. The taper allows the stylet to accommodate multiple OETT sizes. The ridges engage the OETT in a firm, secure grip, giving the user control of the OETT during the air-Q Blocker removal process. The grooves allow spontaneously breathing patients unimpeded air passage within the OETT during removal of the air-Q Blocker. By immobilizing and exerting an inward stabilizing force on the OETT, the air-Q Blocker Removal Stylet allows for the swift removal of the air-Q Blocker without dislodging the previously-placed OETT from the patient.

- Remove the OETT connector from the OETT.
- Squeeze the proximal portion of the OETT between the index finger and the thumb, leaving enough room for the adapter portion of the stylet to enter the proximal opening of the OETT. Alternatively, squeeze the proximal end of the air-Q Blocker airway tube, trapping the OETT inside.
- Insert the tapered end of the air-Q Blocker Removal Stylet into the

- proximal OETT (the long axis should be in the 12 o'clock - 6 o'clock position) until the adapter fits snugly within the OETT.
- For larger sizes, (2.0 - 4.5), with firm inward pressure, rotate the stylet adapter in a clockwise direction (into the 3 o'clock - 9 o'clock position) until the adapter firmly engages the OETT. For smaller sizes (1.0-1.5) simply push the stylet firmly into the OETT. Please practice this a few times prior to attempting on a patient.
- Completely deflate the air-Q Blocker cuff and pilot balloon.
- Deflate and lubricate the pilot balloon on the OETT prior to withdrawing the air-Q Blocker. Reinflate the OETT following air-Q Blocker removal.
- While exerting an inward stabilizing force on the stylet, slowly withdraw the air-Q Blocker outward over the stylet rod.
- For larger sizes (2.0-4.5) pass the stylet through and through. For smaller sizes (1.0-1.5) remove the stylet from the proximal end of the air-Q Blocker while stabilizing the OETT at the mouth. Following removal, place the reusable air-Q Blocker into a suitable container for sterilization and reuse. Discard single-use air-Q Blocker's following use.
- Reposition the OETT to the proper depth within the patient, if needed, and then tape into place.
- Replace the OETT connector within the OETT. Inflate the OETT, if needed, and attach to an appropriate breathing device. Check for adequate ventilation.

Cautions/Warnings


- Inspect all air-Q Blocker devices prior to use. Discard defective devices.
- Do not use sharp instruments on or near the air-Q Blocker.
- Confirm that the air-Q Blocker size matches the connector size prior to use.
- Confirm complete connector engagement within the airway tube prior to use.
- The air-Q Blocker® connector is removable and airway disconnect is possible. Take standard precautions to minimize the chance of disconnect.
- Do not use excessive force during air-Q Blocker placement or removal.
- Immediately check for adequate ventilation following placement.
- If airway problems occur and persist, remove the air-Q Blocker and establish an effective airway by another method. Back-up means for ventilation should be readily available.
- Deflate the air-Q Blocker cuff and pilot balloon completely prior to removal.
- air-Q Blocker connectors may dislodge during use following lubrication. Clean the breathing tube and connector thoroughly with alcohol prior to re-use.
- air-Q Blocker single-use airways are constructed using Di-isononyl phthalate (DINP), air-Q Blocker airways are designed for short term oral use which is not considered part of the known risk group for DINP which includes hemodialysis, blood transfusion, and extra cor- poreal oxygenation. Risk benefits will be evaluated on a per case basis.
- Maximum air-Q Blocker cuff pressure 60cm H₂O. Cuff volume and/or pressure may change with the use of nitrous oxide or other medical gases. **DO NOT OVERINFLATE.**
- Supralaryngeal Airways, including the air-Q Blocker, do not fully protect the patient from aspiration.
- Recheck airway position and patency following all changes in the patient's head or neck position.
- Supralaryngeal Airways are potentially flammable in the presence of lasers and electrical cautery.
- Placement and maintenance of a bite block is recommended during air-Q Blocker use.
- Re-use of single-use devices may lead to mechanical malfunction and potential micro biological contamination.
- Discard all single-use air-Q Blocker's following use.**
- The single-use air-Q Blocker has been sterilized utilizing Ethylene Oxide, a known carcinogen.

Contraindications

The air-Q Blocker is contraindicated in patients at high risk for regurgitation and/or aspiration. This includes, but is not limited to, patients undergoing major thoracic or abdominal surgery, patients who are non-fasted, morbidly obese, pregnant > 14 weeks, or suffer from delayed gastric emptying or esophageal reflux. Users must weigh the benefits of emergency airway needs with the potential risk of aspiration in these patients. air-Q Blocker's should be used in unconscious or topically anesthetized patients only.

Adverse Effects

Previously reported adverse events with masked laryngeal airways include: sore throat, aspiration, regurgitation, vomiting, bronchospasm, gagging, hiccup, coughing, transient glottic closure, airway obstruction, laryngeal spasm, retching, breath holding, arytenoid dislocation, trauma and/or abrasion to the epiglottis, larynx, pharynx, uvula, hyoid and tonsils, tongue cyanosis, lingual nerve, vocal cord and hypoglossal nerve paralysis, tongue macroglossia, parotid gland swelling, dry mouth, dysphagia, feeling of fullness, mouth ulcer, dysarthria, dysphonia, hoarseness, stridor, pharyngeal ulcer, pulmonary edema, laryngeal hematoma, head and neck edema, myocardial ischemia and dysrhythmia.

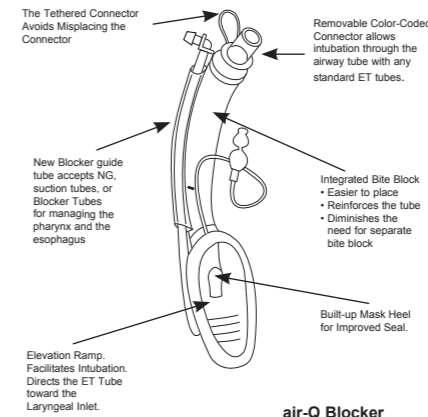air-Q Blocker

Discard all defective air-Q Blocker's.

Warranties

Cookgas® LLC agrees to warrant the disposable air-Q Blocker for a period of 30 days following the invoice date. Warranty covers materials and manufacturing defects provided that the airway is used according to the procedures outlined in the Instructions For Use (IFU) manual. Warranty is valid only following purchase from authorized distributors.

The disposable package label must accompany the defective air-Q Blocker device for valid warranty returns.

Cookgas® LLC disclaims all other warranties whether expressed or implied.

Distributed exclusively by:

Salter Labs®

For Ordering Information, Contact:

30 Spur Drive
El Paso, Texas USA
Telephone: 800-421-0024
www.salterlabs.com

Patents U.S	Patents CAN	Patents UK
5,937,860	2,231,331	GB2324040B
US 6,705,321 B2	2,371,435	GB2407293B
US 7,357,845 B2		GB2405589B
US 7,780,900 B2		GB2357437B
US 7,784,464 B2		
US 7,900,632 B2		

Other USA & Foreign Patents Pending

Do Not Reesterilize (Disposable Only) **EC REP**

M T Promedt Consulting
Altenhofstrasse 80 • D-66386 St. Ingbert
Germany
Tel. +49(0).6894.581020 • Fax +49(0).6894.581021
email: info@mt-procons.com
www.mt-procons.com

0482


Recommended insertion Technique

Anbefalet indføringsteknik
 Aanbevolen inbrengtechniek
 Technique d'insertion conseillée
 Empfohlene Technik zur Einführung
 Συνιστώμενη τεχνική εισαγωγής
 Tecnica di inserimento consigliata
 Anbefalt innsettsingsmetode
 Zalecana technika wprowadzania
 Técnica de inserção recomendada
 Técnica de inserción recomendada
 Rekommenderad insättningssteknik
 Tavsiye Edilen Yerleşirme Tekniği

Recommended Depth of Insertion Range

Anbefalet indføringsdybde
 Aanbevolen inbrengdiepte-bereik
 Plage de profondeur d'insertion conseillée
 Empfohlener Einführtiefenbereich
 Συνιστώμενο βάθος εύρους εισαγωγής
 Profondità dell'intervallo di inserimento consigliata
 Anbefalt dybde i innsettsingsområde
 Zalecany zakres głębokości wprowadzenia
 Profundidade recomendada do intervalo de inserção
 Intervalo de profundidad de inserción recomendado
 Rekommenderat djup i insättningsområdet
 Tavsiye Edilen Yerleşirme Derinliği Aralığı

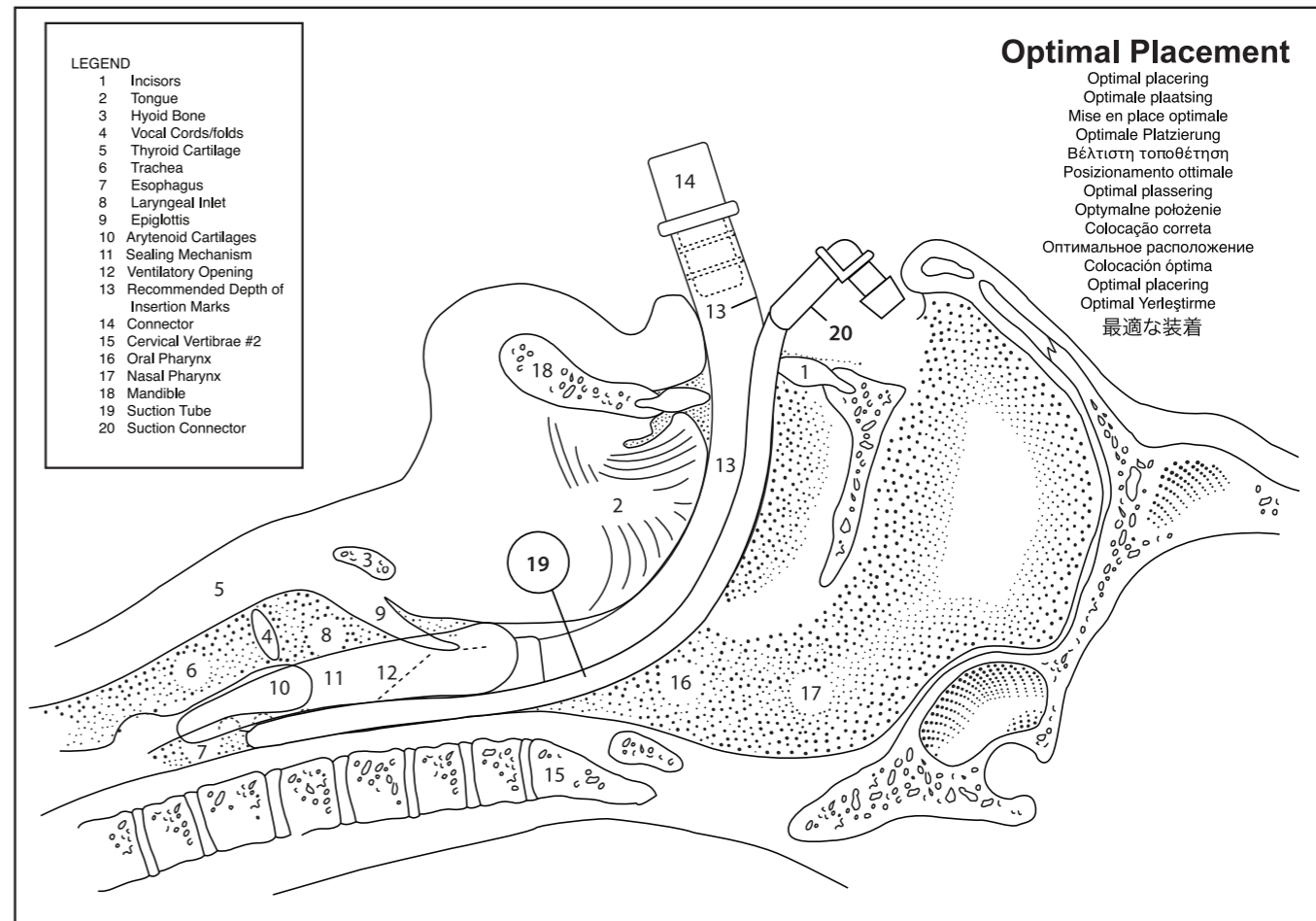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Cookgas®, LLC rechaza cualquier otra garantía explícita o implícita.