EN: ENGLISH

Aspen Full Face Mask

Instructions for Use – 11016 Rev C

Intended Use

The Aspen is intended for use by adults (> 30 kg) as a patient interface for CPAP or Bi-Level positive-pressure ventilation devices for treatment of respiratory insufficiencies and obstructive sleep apnea. The Aspen may be reused multiple times by a single patient.

Storage Temperature:
-20 to 60°C
(-4 to 140°F)

The deadspace associated with this device is less than 150 cm³.

Warnings

- In the USA, federal law restricts this mask to sale by, or on the order of, a physician. (Rx ONLY)
- The device should be used only with the ventilation system recommended by your physician or respiratory therapist. A mask should not be used unless the ventilation system is turned on and operating properly. The exhalation ports should never be blocked.
 - Explanation of warning:* Ventilation systems are intended to be used with special masks with exhalation ports to allow a continuous flow of air out of the mask. When the ventilation system is turned on and functioning properly, new air from the ventilation system flushes the exhaled air out through the exhalation ports. However, when the ventilation system is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation. This warning applies to most models of ventilation systems.
- If you experience any reaction or notice any redness or irritation of your skin, discontinue use and contact your physician. The device consists of silicone, polycarbonate, polypropylene, nylon, polyester, and neoprene.
- If oxygen is used with the ventilation system, the oxygen flow must be turned off when the ventilation system is not operating. *Explanation of the warning:* When the ventilation system is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the ventilation system enclosure. Oxygen accumulated in the ventilation system enclosure will create a risk of fire. This warning applies to most types of ventilation systems. Do not smoke while oxygen is in use. Note: At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate. This warning applies to most types of ventilation systems.
- At low CPAP pressures, the flow through the exhalation ports may be inadequate to clear all exhaled gas from the tubing. Some rebreathing may occur.
- A minimum pressure of 3.0 cm H2O at the mask is required.
- The anti-asphyxia valve allows the patient to breathe room air in the case that the CPAP/Bi-level device connected to the mask is not operating or is turned "off". The valve closes at operating pressures of 3.0 cm H2O and above. The mask should not be used if the anti-asphyxia valve is missing, damaged, or not working properly. The large opening at the valve should never be blocked.
- Do not use the mask if vomiting or feeling nauseous. Failure to arouse and remove the mask after vomiting could result in aspiration of vomitus. To minimize the risk of vomiting during sleep, avoid eating or drinking for three hours prior to sleep. This mask is not recommended if the patient is taking a prescription drug that may cause vomiting.
- This device is intended for single-patient use only and thus use by multiple patients is to be avoided due to the risk of transmitting infection. Do not attempt to sterilize or use disinfectants on this device or any of its components.
- The mask may deactivate the pressure alarm on some ventilation systems. Confirm that your ventilation system pressure alarm works with the product before using the pressure alarm.
- Any unusual chest discomfort, shortness of breath, stomach distention, belching or severe headache during or immediately after use should be immediately reported to your physician.

Contraindications


This mask may not be suitable for persons with the following conditions: impaired cardiac sphincter function, excessive reflux, impaired cough reflex, and hiatal hernia. It should not be used if patient is uncooperative, obtunded, unresponsive, or unable to remove the mask.

Assembling the Device

- Prior to use, clean the mask as described in the cleaning instructions.
- Press the cushion completely into the shell.
- Attach the headgear to the forehead support by inserting the two loose straps into the slots on the forehead support.

Fitting the Device

- Clean and thoroughly dry your face before putting on the mask.
- Attach the circuit tube of the ventilation system to the elbow.
- Inspect the anti-asphyxia valve located within the clear elbow. With the CPAP or bi-level system turned off, verify that the valve is positioned so that room air can flow through the large opening. Turn on the system; the valve should close the large opening and system air should flow into the mask.
- Place the mask over the mouth and position the cushion flap under the chin. Pull the headgear back over the top of the head.
- Attach the two lower buckles of the headgear to the cylindrical portion of the posts on the mask and pull the buckles back (towards the face) until an audible "click" is heard.
- Unhook the hook-and-loop closures of each of the headgear straps in pairs (top pair and bottom pair), gently pull until the mask sits squarely on your face, and then reconnect the hooks. The mask should fit snug enough to prevent large leaks yet still be comfortable. In general, a looser fit is better. Over-tightening may actually make leaks worse and create discomfort. Next, adjust the over-head straps as necessary.
- Turn on your ventilation system.
- Breathe normally, and continue making minor adjustments until you have a comfortable fit with minimal leaks. If you are unable to achieve a comfortable fit or to resolve any leaks, you may need an alternative cushion size.
- To remove the mask, you do not need to unhook each strap. Just unsnap the two lower buckles of the headgear from the mask, and then lift the mask off over the head.

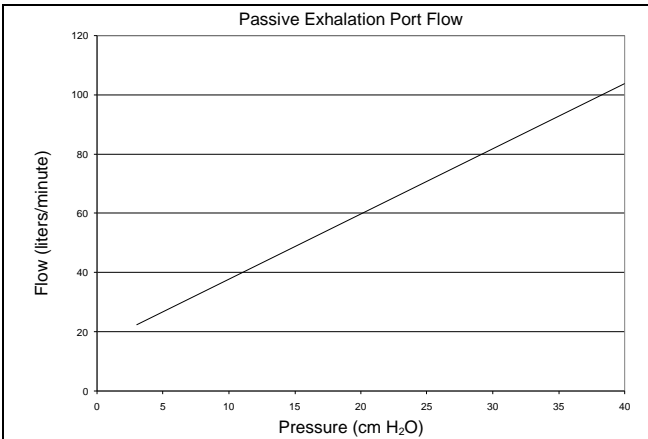Cleaning Instructions

After Each Use:

- Remove the fabric parts from the mask as the fabric parts will be washed separately. The fabric parts should be handwashed following the remaining instructions at least every 7 days. Do not machine wash.
- Remove the cushion from the shell.
- Handwash all components in warm, soapy water. NOTE: Only use pure soap and do not use alcohol, petroleum products, or strong cleaners as these may cause damage and reduce the life of the product.
- Make sure that the exhalation ports on the elbow are free of any obstructions.
- Rinse thoroughly and shake excess water from interior surfaces of components. Allow all components to completely air-dry, away from direct sunlight.
- Carefully inspect each component for any wear or damage. Discontinue use if you notice any cracks in the plastic, any tears or hardening of the silicone, or significant fraying or loss of elasticity of the headgear, and contact your healthcare provider for a replacement. Discoloration is normal after several washings and should not be considered sign of wear.

Manufactured for:



30 Spur Dr., El Paso – TX 79906 – USA
Tel: (800) 421-0024 – www.salterlabs.com

Covered by patents: U.S. D687538. Other patents pending.

Reorder Information

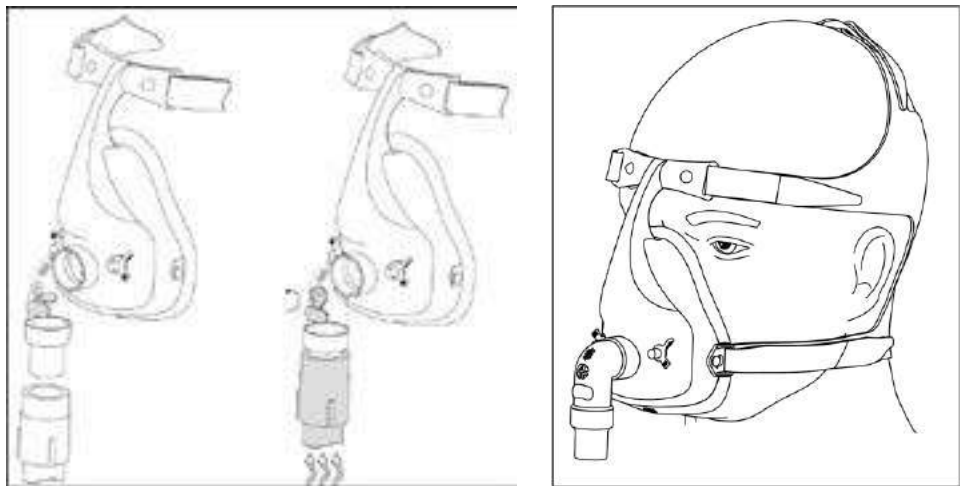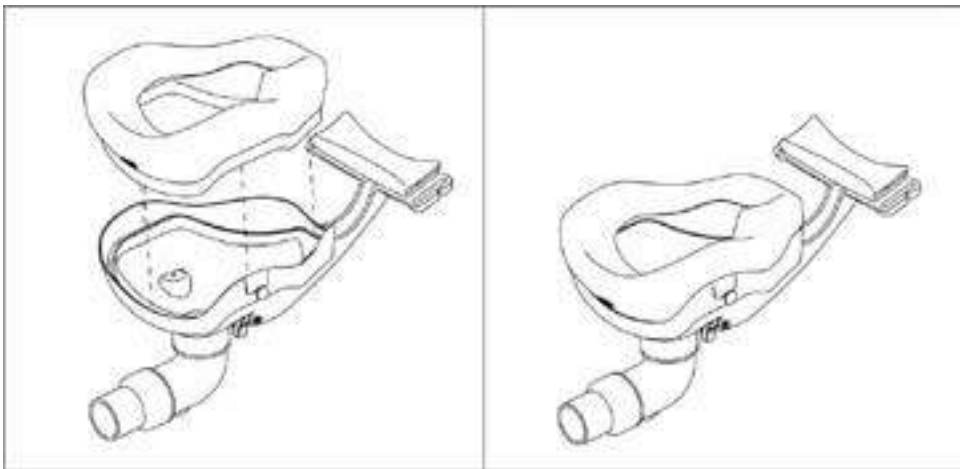Part Number	Description	Part Number	Description
ASP201	Aspen – SM (<i>Small</i>)	ASP211	Cushion, <i>Small</i>
ASP203	Aspen – MD (<i>Medium</i>)	ASP213	Cushion, <i>Medium</i>
ASP205	Aspen – LG (<i>Large</i>)	ASP215	Cushion, <i>Large</i>
		ASP230	Headgear

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FoxxMed Ltd.