
Nasal-Aire® II Petite

LOT _____

Passive Exhalation Port Flow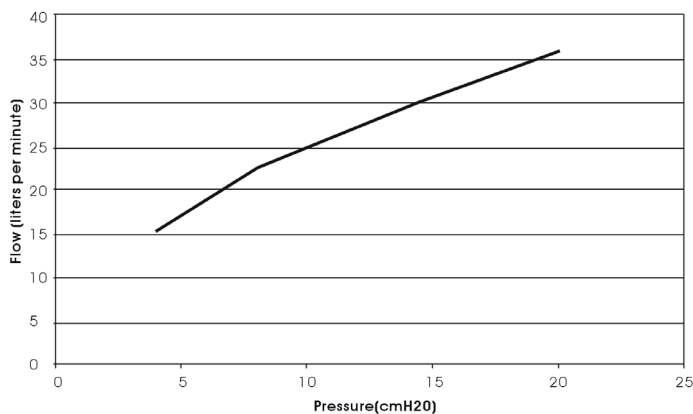

Covered by Pat. No. 6,595,215 and Patents Pending

ENGLISH

Homecare Applications

The Nasal-Aire II Petite CPAP Interface with L-Flow™ technology is an accessory to positive pressure ventilation devices for patients with Obstructive Sleep Apnea requiring positive pressure breathing therapy (i.e. CPAP, BiLevel). The Nasal-Aire II Petite is intended for adult patients (> 30 kg). For homecare applications, this interface may be reused multiple times by a single patient, with proper cleaning.

Nasal-Aire II Petite is designed for patient comfort and to facilitate patient compliance, so that the maximum benefit of the prescribed therapy may be derived.

System Components

Nasal-Aire II Petite consists of a soft nosepiece, which has exhaust ports and two soft silicone nasal cannulas for fitting into the nasal openings. Nasal-Aire II Petite also consists of tubing, connectors, headgear, and a "Y"-shaped swivel coupling. At the base of the swivel coupling is a rigid port for connection of the Nasal-Aire II Petite to a positive pressure ventilation device.

Preparation for Use

1. Remove Nasal-Aire II Petite assembly from package.
2. Wash and rinse assembly according to the Cleaning Instructions before use.
3. Complete circuit assembly before use (see illustration "ASSEMBLY").

Directions for Use

1. Pull headgear forward over top of head and adjust headgear for comfort.
2. Grasp the cannula nosepiece with thumbs and fingers under each nasal insert and position near nostrils.
3. Using thumbs, press inserts into nostrils until base of Nasal-Aire II Petite nosepiece is flush against the nose.
4. Insert the left holder strap of the headgear through the left loop on the cannula. Insert the right holder strap of the headgear through the right loop on the cannula.
5. Adjust location of the holder straps on the headgear as needed for comfort.
6. Attach CPAP tubing to swivel coupling and turn CPAP machine on.
7. Once airflow starts, adjust Nasal-Aire II Petite to fit for comfort and seal. Periodic adjustments may be necessary.

Cleaning Instructions

1. Disconnect nasal cannula from tubing connectors. Do not disassembly other components.
2. Clean all components with mild soap and warm water. Soaking for extended periods of time is not recommended.
3. Shake excess water from interior surfaces of components.
4. A cotton swab may be used to clean openings of tubing, nosepiece and nasal inserts.
5. Set aside to air dry.
6. Reassemble before use. (see illustration "ASSEMBLY").

Nasal-Aire II Petite should be cleaned after each use. Testing has shown that the Nasal-Aire II Petite may be cleaned at least ninety (90) times. Discard when signs of wear are evident. Signs of wear include rips, tears, holes to the cannula or tubing, or visible cracking of any component. Discoloration is normal after several washings, and should not be considered a sign of wear. See your homecare provider for replacement.

Warnings and Precautions

- Federal (USA) law restricts this device to use by or on order of a physician. Use only as prescribed by a qualified physician.

- Do not use this device without instruction from your doctor, home health care professional, or your respiratory therapist.
- Nasal-Aire II Petite must be used only with a positive pressure ventilation device.
- Nasal-Aire II Petite is intended for single-patient use only. Use by multiple patients is to be avoided due to the risk of transmitting infection.
- For homecare applications, do not attempt to sterilize or use disinfectants on this device or any of its components.
- Nasal-Aire II Petite is intended for adult patients only (> 30 kg).
- The Nasal-Aire II Petite should not be used unless the ventilation system is turned on and operating. The exhalation ports should never be blocked.

Explanation of warning: CPAP systems are intended to be used with special nasal cannulas with intentional ports to allow a continuous flow of air out of the nasal cannula. When the CPAP machine is turned on and functioning properly, new air from the CPAP machine flushes the exhaled air through the intentional leak port. However, when the CPAP is not operating, enough fresh air will not be provided through the nasal cannula, and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can in some cases lead to suffocation. This warning applies to most models of CPAP machines.

- Nasal-Aire II Petite is not intended for use on ventilation systems with pressure alarms. The device can deactivate the alarm.
- Some users may experience skin redness. If this occurs, consult your doctor.
- Do not expose this device to extremes of temperature. Store the Nasal-Aire II Petite at temperatures between 0°C (32°F) and 54°C (130°F).

Please direct any questions concerning your Nasal-Aire II Petite accessory to your healthcare professional OR Innomed Technologies at www.innomedinc.com or to technical support at 1-800-200-9842. Obtain additional technical specifications from InnoMed Technologies technical support at 1-800-200-9842.

Reorder information

Nasal-Aire II Petite Sizes	Complete System	Cannula Only	Headgear	Sizing Gauge
Petite A (largest)	PA401	PA411	P430	X4007
Petite B	PB402	PB412		
Petite C	PC403	PC413		
Petite D	PD404	PD414		
Petite E (smallest)	PE405	PE415		

NOTE: Retain these instructions and the lot number of your Nasal-Aire II Petite. Please reference the lot number if you need to contact InnoMed Technologies or your homecare provider about your Nasal-Aire II Petite.

Nasal-Aire II Petite is brought to you by:

Manufactured for: Salter Labs
30 Spur Dr., El Paso, TX 79906 USA
Tel: (800) 421-0024 www.salterlabs.com

Manufactured by: MODERN MEDICAL EQUIPMENT MFG. LTD.
5fl, 35 Tai Lin Pai Rd., Kwai Chung, Hong Kong.
Tel: (852) 2420 9068 Fax: (852) 2481 1234

European Representative: Emergo Europe
Molenstraat 15, 2513 BH The Hague, The Netherlands.
Email: mmeq_eu@modernmedical.com

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Valmistaja: MODERN MEDICAL EQUIPMENT MFG. LTD.