

REF 0205

VixOne™ Reusable Nebulizer

QTY: 1

Vixone Reusable Nebulizer

Individually Bagged

Not made with Natural Rubber Latex

Not made with DEHP



LATEX FREE

SUGGESTED PROCEDURE:

1. These products are intended for use in delivering aerosol medications.
2. All products are to be used in accordance with the applicable prescription.
3. The VixOne is packaged preassembled. Medication may be added directly through the opening in the top of the Nebulizer (see reverse side for assembly).
4. If preferred, the Nebulizer components may be separated to facilitate the addition of medication. To open the Nebulizer, grip the Top, rotate the Bowl slightly until the Top is released, and pull to separate.
5. Add medication to the Nebulizer Bowl.
6. Reassemble the Nebulizer as follows: Align the Nebulizer Top with the Bowl, press the components together firmly, and then rotate the Bowl until it locks in place. Confirm that the Top and Bowl are completely engaged to ensure optimum performance.
7. Configure Nebulizer for application by attaching prescribed auxiliary components.
8. Only the VixOne nebulizer is top shelf dishwasher safe.
9. See reverse side for cleaning instructions.

CAUTION: Do not over tighten: Stripping of the locking mechanism may occur.

Westmed 

5580 S. Nogales Highway
Tucson, Arizona 85706 USA

Phone: 800-975-7987

Fax: 520-294-6061

www.westmedinc.com

EC

REP

MT Promedt Consulting GmbH
Altenhofstr. 80

66386 St. Ingbert, Germany
ASSEMBLED IN MEXICO

LABEL PN 700251, Rev.01

LOT

Assembly Illustration

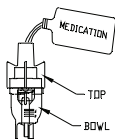


FIGURE 1

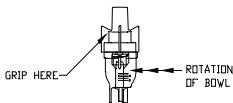


FIGURE 2

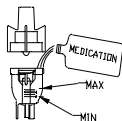


FIGURE 3

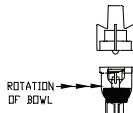


FIGURE 4



FIGURE 5

Cleaning Instructions:

The VixOne™ Nebulizer must be disassembled and disinfected periodically in order to operate correctly. Westmed recommends full disassembly and disinfection of all components at the end of each treatment day.

1. Wash and dry your hands.
2. Disassemble the nebulizer.
3. Wash all parts except tubing and mask (if applicable) in warm water and liquid dishwashing soap, or soak all parts in a solution of 1 part household bleach and 50 parts water for 3 minutes. Rinse Thoroughly.
4. Place all parts on a clean surface (fresh paper towel) to dry.
5. After parts are dry, reassemble and store kit until next use.

Please discuss the various methods of aerosol administration and proper cleaning techniques with your medical provider or respiratory home care company.



CAUTION:

Do Not Use Vinegar. Do not boil or microwave as high temperatures may melt some of the plastic parts.