

REF 0383**UNITS - 1****RX Only**

Pedi-Neb™ Nasal Nebulizer
Nebulizador Nasal **Pedi-Neb™**
Nebulizador Nasal **Pedi-Neb™**
Nébuliseur Nasal **Pedi-Neb™**
Nebulizzatore Nasale **Pedi-Neb™**
Pedi-Neb™ Nasalvernebler
w/ Extension Kit

These products are intended for use in delivering aerosol medications to infants.
All products are to be used in accordance with the applicable prescription.

SUGGESTED USE:

1. Assess patient. Nasal passages must be clear. Follow clinical procedure to clear nasal passages.
2. Remove the Nebulizer from the package and add the prescribed medication.
3. Assemble the Nebulizer per the product illustration.
4. Attach the Nebulizer to the components included in this kit and then connect the supply tube to the gas source. Rotate the Threaded Nut (if included) 1/4 turn to ensure proper fit.
5. Ensure that all connections are secure, and then start the gas flow.
6. Confirm appropriate aerosol function.
7. Discard after single patient use.

CAUTION: TO PREVENT POSSIBLE CHOKING HAZARD, check the silicon nipple portion by pulling hard on all parts. If Pedi-Neb™ becomes sticky, swollen, cracked, torn or shows other signs of wear, DISCONTINUE USE IMMEDIATELY. Do not leave children unattended with pacifier.

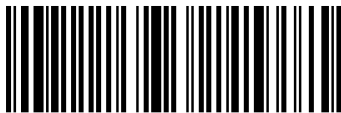
Westmed

5580 S. Nogales Highway
Tucson, Arizona 85706 USA
Phone: 800-975-7987
Fax: 520-294-6061
www.westmedinc.com



MT Promedt Consulting GmbH
Altenhofstr. 80
66386 St. Ingbert, Germany

Made in USA
LABEL PN 70836, Rev. 09

LOT

(01)00709078001598



(10)LLLLLL

