

REF 0101

UNITS - 01

Rx Only



BIFLO™ MASK
MASCARILLA BIFLO™
MÁSCARA BIFLO™
MASQUE BIFLO™
MASCHERA BIFLO™
BIFLO™ MASKE



CAUTION:

Use of sterilization or cleaning solutions may leave this device nonfunctional.

SUGGESTED PROCEDURE:

1. These products are intended for use in delivering oxygen therapy.
2. All products are to be used in accordance with the applicable prescription.
3. Remove the product from the package.
4. Attach the device to the oxygen source and make appropriate connection.
5. Ensure that all connections are secure, then start the gas flow.
6. Confirm appropriate function prior to use.
7. Discard after single patient use.

Westmed

5580 S. Nogales Highway
Tucson, Arizona 85706 USA
Phone: 800-975-7987
Fax: 520-294-6061
www.westmedinc.com



MT Promedt Consulting GmbH
Altenhofstr. 80
66386 St. Ingbert, Germany

LOT

Patents Pending
ASSEMBLED IN MEXICO
LABEL PN 71018, Rev. 06