

REF 0507

UNITS - 1

Rx only

ADULT DEMAND CANNULA

Adult nasal cannula w/4' oxygen delivery tubing, for use with Dual Port Demand Valve



SUGGESTED PROCEDURE:

1. Position cannula by inserting tips into nostrils.
2. Thread cannula tubing over ears and adjust slide upwards for a comfortable fit under the chin.
3. Attach tubing connectors to dual port demand unit.
4. Set oxygen to the prescribed liter flow.



WARNING:

1. Refer to demand oxygen delivering unit recommendations prior to use.
2. Do not use on patients with obstructions in one or both nostrils.

Westmed

5580 S. NOGALES HWY.
TUCSON, AZ 85706 USA
Phone: 1-800-975-7987
Fax: 520-294-6061
www.westmedinc.com
US Patent Pending

EC

REP

MT Promedt Consulting GmbH
Altenhofstr. 80
66386 St. Ingbert, Germany

ASSEMBLED IN MEXICO

LOT

Label PN 71520, Rev. 05