

REF 0430

Comfort Soft Plus™

ORAL/NASAL SAMPLING CANNULA

Adult Oral/Nasal Cannula, 10' O₂/CO₂ Tubing
w/Male Luer, 2" Pigtail w/Female Luers

Not Made with Natural rubber latex

Not made with DEHP

QTY- 1

Rx only



Contains One Each Per Package • Disposable Non-Sterile • Clean Ready for Use

CAUTION:

- Do not sterilize.

SUGGESTED PROCEDURE:

1. Position cannula by inserting tips into nostrils.
2. Thread cannula tubing over ears and adjust slide upwards for a comfortable fit under the chin.
3. Attach oxygen supply tube to oxygen source and luer connector to CO₂ monitor.
4. Set oxygen to the prescribed liter flow.
5. Check for gas flow throughout the device.
6. Check positioning on device to assure proper detection of CO₂.

WARNING:

1. Accuracy of CO₂ monitoring can be diminished if the patient has a perforated septum.
2. Do not use on patients with obstructions in one or both nostrils.

Westmed 

5580 S. Nogales Highway
Tucson, Arizona 85706 USA

Phone: 800-975-7987

Fax: 520-294-6061

www.westmedinc.com

US Patent Pending

LOT

EC

REP

MT Promedt Consulting GmbH

Altenhofstr. 80

66386 St. Ingbert, Germany

Assembled in Mexico

Label PN 71946 Rev. 03