

REF 0413

QTY - 1

Rx Only

DISPOSABLE MEDICAL HYPERINFLATION SYSTEM

Transport Elbow, 8' Oxygen Tubing
with Threaded Nut



Contents: The contents are packaged per the Catalog Number noted above.

PLEASE READ THE SUGGESTED DIRECTIONS FOR USE AND/OR WARNINGS BELOW.

INTENDED USE: This device is intended for use with devices where continuous gas flow is directed to the patient. When used as directed, this device is designed to bleed off excess supply gas which could result in undesired PEEP. This device is not intended to be used in place of a pressure limiting valve unless used in conjunction with a pressure monitoring gauge.

SUGGESTED PROCEDURE:

1. Secure all connections.
2. Attach oxygen supply tube to oxygen flowmeter.
3. Attach manometer (not included) to pressure port located next to adjustment knob on hyperinflation elbow.
4. Adjust oxygen flow rate to desired flow rate based on clinical judgment.
5. Turn relief valve adjustment knob to fully closed position.
6. Occlude patient port on elbow by placing thumb over 15 mm port opening and allow bag to inflate. Check for proper operation.
7. With elbow still occluded, check flowmeter and pressure adjusting knob for desired flow and pressure based on clinical experience and judgment.
8. Attach to patient adaptor. As bag inflates, initiate ventilation in accordance with recognized procedures. If bag starts to over-inflate, reduce gas flow and/or increase pressure adjusting knob opening.

CAUTION: This device should only be used in conjunction with a manometer (if using a hyper inflation set) to determine appropriate delivery pressure. This device is designed to be used by medical personnel familiar with ventilation and airway maintenance techniques.

WARNING: Do not use with flammable gases. Do not soak, rinse, wash, gas or steam sterilize these products. To do so may result in harmful residual gases, or dangerously increase its breathing resistance.

Westmed 

5580 S. Nogales Highway
Tucson, Arizona 85706 USA
Phone: 800-975-7987
Fax: 520-294-6061
www.westmedinc.com

LOT

Assembled in Mexico

LABEL PN 74812, Rev. 05