

REF 0557

UNITS - 1

LENGTH - 2"

Comfort Soft Plus[®] Cannula High Flow

Premature Nasal Cannula for use with flows up to 7 LPM.
Yellow Band on Cannula

Rx Only



Not Made
With Natural
Rubber Latex



Not Made
With DEHP

CE 0482



CAUTION:

Use of sterilization or cleaning solutions
may leave this device nonfunctional.

SUGGESTED PROCEDURE:

1. These products are intended for use in delivering oxygen therapy.
2. All products are to be used in accordance with the applicable prescription.
3. Remove the product from the package.
4. Attach the device to the oxygen source and make appropriate connection.
5. **Refer to instructions on back side for connection to circuit.**
6. Ensure that all connections are secure, and then start the gas flow.
7. Confirm appropriate function prior to use.
8. Check for gas flow throughout the device.
9. Discard after single patient use.

LOT

Westmed 

5580 S. Nogales Highway
Tucson, Arizona 85706 USA

Phone: 800-975-7987

Fax: 520-294-6061

www.westmedinc.com

EC

REP

MT Promedt Consulting GmbH

Altenhofstr. 80

66386 St. Ingbert, Germany

Patents Pending

Product of Mexico

Assembled in Mexico

LABEL PN 75506, Rev. 01

SUGGESTED USE:

1. Attach appropriate Westmed Cannula Adapter.
2. Connect small end of Adapter to High Flow Cannula.
3. Connect large end of Adapter to Circuit.
4. Prior to patient connection: Check that all connections are secure and confirm there is not leakage or occlusion of the system.

! **WARNING:** Do not soak, rinse, wash, gas, or steam sterilize this product.

