

REF 0415

QTY: 1

LENGTH: 60" (1.5m)

Neo-Vent™

INFANT T-PIECE

RESUSCITATION CIRCUIT

Rx Only



**Circuit for use on compatible Infant Resuscitators.
15mm OD X 10mm ID connector**

Product is intended for use in delivering oxygen therapy.

Product is to be used in accordance with the applicable prescription.

SUGGESTED PROCEDURE:

1. Remove product from package.
2. **Ensure that all connections are secure.**
3. Set PIP and PEEP following Infant Resuscitators Instructions For Use.
4. Recommended operating gas flow range is 5 to 15 L/min. Do not attempt to use a higher flow rate than 15 L/min.
5. Confirm appropriate function.

Westmed

5580 S. Nogales Highway
Tucson, Arizona 85706 USA
Phone: 800-975-7987
Fax: 520-294-6061
www.westmedinc.com

EC

REP

MT Promedt Consulting GmbH
Altenhofstr. 80
66386 St. Ingbert, Germany

LOT

Assembled in Mexico
LABEL PN 76126, Rev. 04