

**REF 0550**

**UNITS- 1**

**LENGTH - 7' (2.1 m)**

**Rx Only**

**Comfort Soft Plus®**

**ADULT CANNULA HIGH FLOW**

CANULA DE ALTO FLUJO PARA ADULTO

CANULE ADULTE À HAUT DÉBIT

HOCHDURCHFLOSS-KANÜLE, ERWACHSENE

ADULTO CANULA DE ALTO FLUXO


CANNULA ADULTI AD ALTO FLUSSO



Not Made  
With DEHP



Not Made  
With Natural  
Rubber Latex

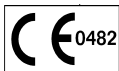

**SUGGESTED PROCEDURE:**

1. These products are intended for use in delivering oxygen therapy.
2. All products are to be used in accordance with the applicable prescription.
3. Remove the product from the package.
4. Attach the device to the oxygen source and make appropriate connection.
5. Ensure that all connections are secure, and then start the gas flow.
6. Confirm appropriate function prior to use.
7. Check for gas flow throughout the device.
8. Discard after single patient use.

**⚠ CAUTION:** Use of sterilization or cleaning solutions may leave this device nonfunctional.

**NOTE:** For use with individual concentrators with pressure and flows from 1 to 10 LPM. Wall or compressed gas source at flows flows from 1 to 15 LPM.



MT Promedt Consulting GmbH  
Altenhofstr. 80  
66386 St. Ingbert, Germany

**Westmed** 

5580 S. Nogales Highway  
Tucson, Arizona 85706 USA

Phone: 800-975-7987

Fax: 520-294-6061

[www.westmedinc.com](http://www.westmedinc.com)

Patent Pending

Assembled in Mexico

LABEL PN 77588, Rev. 01