

Westmed, Inc.

5580 S. Nogales Highway

● Tucson, AZ 85706 USA

● (800-975-7987)

Rx Only



IVD

STERILE R


IFU: Blunt Tip Cannula w/Collar

For non-venting syringe:

1. Remove luer tip cap. Set aside for future use.
2. Depress plunger to bottom of syringe.
3. Prepare arterial line for blood sampling.
4. Attach syringe to arterial line by pushing blunt cannula onto sampling port.
5. Pull plunger back to aspirate desired sample volume.
6. Remove from port. Secure arterial line.
7. Remove blunt cannula. Attach luer tip cap. Mix sample.

For venting syringe:

1. Preset plunger to desired sample volume.
2. Prepare arterial line for blood sampling.
3. Attach syringe to arterial line by pushing blunt cannula onto sampling port.
4. Open arterial line and allow syringe to fill desired sample volume.
5. Remove from port. Secure arterial line.
6. Remove blunt cannula from sampling port.
7. Remove blunt cannula. Attach luer tip cap. Mix sample.

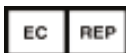

MT Promedt Consulting GmbH
Altenhofstr. 80
66386 St. Ingbert, Germany



5580 S. Nogales Highway
Tucson, Arizona 85706 USA
Phone: 800-975-7987
Fax: 520-294-6061
www.westmedinc.com

Made in Mexico
Assembled in Mexico
Label PN 77639, Rev. 02