

REF 0250

UNITS - 01

Rx Only



CE 0482



Not Made
With DEHP



Not Made
With Natural
Rubber Latex


**AEROSOL MASKS
MASCARILLAS PARA AEROSOL
MÁSCARAS DE AEROSOL
MASQUES POUR AÉROSOL
MASCHERE PER AEROSOL
AEROSOLMASKEN
ADULT, ADULTOS, ADULTO,
ADLTE, ADULTI, ERWACHSENE**

SUGGESTED PROCEDURE:

1. These products are intended for use in delivering aerosol medications.
2. All products are to be used in accordance with the applicable prescription.
3. Remove the Nebulizer from the package and add the prescribed medication.
4. Assemble the Nebulizer Top to the Bowl per the Instruction Sheet.
5. Attach the Nebulizer to the components included in this kit and then connect the supply tube to the gas source. Rotate the Threaded Nut (if included) 1/4 turn to ensure proper fit.
6. Ensure that all connections are secure, and then start the gas flow.
7. Confirm appropriate aerosol function.
8. Discard after single patient use.


⚠ CAUTION:

Use of sterilization or cleaning solutions may leave this device nonfunctional.
Store in a cool dry place.

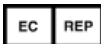

LOT

MT Promedt Consulting GmbH
Altenhofstr. 80
66386 St. Ingbert, Germany

Westmed 

5580 S. Nogales Highway
Tucson, Arizona 85706 USA
Phone: 800-975-7987 - Fax: 520-294-6061
www.westmedinc.com

Product of Mexico
Assembled in Mexico
LABEL PN 77715, Rev. 01