

REF 6219

1 each

Rx Only



Not Made
With Natural
Rubber Latex



Not Made
With DEHP



0482

FILTER

In-line Mini HME w/Sampling Port

22mm OD/15mm ID x 22mm ID/15mm OD

29.77ml Dead space

28.8mg H₂O/L @ VT 500ml moisture output

0.72cm H₂O @ 30LPM resistance

SUGGESTED PROCEDURE:

1. Filters may be fitted at patient end or machine end of circuit.
2. HMEs and HME Filters must be fitted at patient end of circuit.
3. Do not use between the patient and a nebulizer or humidifier.
4. Change the filter as required or as per protocol.

CAUTION: For single patient use only. Use of sterilization or cleaning solutions may leave this device nonfunctional. Store in a cool dry place.

NOTE: The presence of water droplets and/or salt crystals observed in the HME indicates normal function.

Westmed

5580 S. Nogales Highway
Tucson, Arizona 85706 USA

Phone: 800-975-7987

Fax: 520-294-6061

www.westmedinc.com

EC

REP

MT Promedt Consulting GmbH
Altenhofstr. 80

66386 St. Ingbert, Germany

Assembled in MEXICO

LABEL PN 78083 Rev. 02

LOT