

REF 7-9900

UNITS - 1

RX Only



Not Made
With Natural
Rubber Latex



Not Made
With DEHP



DISPOSABLE ANESTHESIA/ RESPIRATORY PRODUCTS

Series 900 Adult Large Anesthesia Mask
MASCHERA PER ADULTI GRANDE
MÁSCARA GRANDE PARA ADULTO
GROSS MASKE FÜR ERWACHSENE
MASQUE ADULTE - GRAND

SUGGESTED USE:

1. Connect to breathing circuit or resuscitation device with appropriate fitting.
2. Pressure check to insure tight system and inspect all fittings to insure secure connection.
3. If equipped with an inflation/deflating valve, insert luer tipped syringe into the valve to inflate/deflate cushion.
4. Hand hold or secure with head strap.
5. Discard after use.

WARNING: Patient should be constantly monitored whenever this device is in use. Do not use in the presence of flammable anesthetics. Do not soak, rinse, wash, gas, or steam sterilize this product. To do so may result in harmful residual gases, or dangerously increase its breathing resistance.

Westmed

5580 S. Nogales Highway
Tucson, Arizona 85706 USA
Phone: 800-975-7987
Fax: 520-294-6061
www.westmedinc.com

EC

REP

MT Promedt Consulting GmbH
Altenhofstr. 80
66386 St. Ingbert, Germany

Assembled in Mexico
LABEL PN 78494, Rev. 01

LOT