

REF 0300

UNITS - 1
RX Only



CAUTION: Use of sterilization or cleaning solutions may leave this device nonfunctional.

SUGGESTED USE:

1. These products are intended for use in delivering aerosol medications.
2. All products are to be used in accordance with the applicable prescription.
3. Remove the Nebulizer from the package and add the prescribed medication.
4. Assemble the Nebulizer Top to the Bowl per the Instruction Sheet.
5. Attach the Nebulizer to the components included in this kit and then connect the supply tube to the gas source. Rotate the Threaded Nut (if included) 4/1 turn to insure proper fit.
6. Insure that all connections are secure, and then start the gas flow.
7. Confirm appropriate aerosol function.
8. Discard after single patient use.

Westmed

5580 S. Nogales Highway
Tucson, Arizona 85706 USA
Phone: 800-975-7987
Fax: 520-294-6061
www.westmedinc.com



MT Promed Consulting GmbH
Altenhofstr. 80
66386 St. Ingbert, Germany
Product of Mexico

Assembled in Mexico
LABEL PN 78582, Rev. 01



LOT

AEROSOL MASKS

MASCARILLAS PARA AEROSOL
MÁSCARAS DE AEROSOL
MASQUES POUR AÉROSOL
MASCHERE PER AEROSOL
AEROSOLMASKEN
ADULT, ADULTOS, ADULTO,
ADULTE, ADULTI, ERWACHSENE