

REF 0503

UNITS - 1

RX Only



C **€0482**

Comfort Soft Plus®
Adult Nasal Cannula, 7' O₂/CO₂
Sampling Line w/Male Luer

SUGGESTED USE:

1. Attach oxygen supply tube to oxygen source and luer connector to CO₂ monitor.
2. Set oxygen to the prescribed liter flow.
3. Check for gas flow throughout the device.
4. Check positioning on device to assure proper detection of CO₂.

WARNING:

1. Accuracy of CO₂ monitoring can be diminished if the patient has a perforated septum or during a time of mouth breathing.
2. Do not use on patients with obstructions in one or both nostrils.

 **CAUTION:** Use of sterilization or cleaning solutions may leave this device nonfunctional.

Westmed 
5580 S. Nogales Highway
Tucson, Arizona 85706 USA
Phone: 800-975-7987
Fax: 520-294-6061
www.westmedinc.com



MT Promedt Consulting GmbH
Altenhofstr. 80
66386 St. Ingbert, Germany

US Patent Pending
Assembled in Mexico
Label PN 78650 Rev. 01



LOT