


REF 260-E

UNITS - 1
RX Only



INFANT AEROSOL MASKS
MASCARILLAS PARA AEROSOL
MÁSCARAS DE AEROSOL
MASQUES POUR AÉROSOL
MASCHERE PER AEROSOL
AEROSOLMASKEN
INFANTIL, INFANTIL, BÉBÉ,
INFANTILE, ZUIGELING

 **CAUTION:** Use of sterilization or cleaning solutions may leave this device nonfunctional.
Store in a cool dry place.

SUGGESTED PROCEDURE:

1. These products are intended for use in delivering aerosol medications.
2. All products are to be used in accordance with the applicable prescription.
3. Remove the Nebulizer from the package and add the prescribed medication.
4. Assemble the Nebulizer Top to the Bowl per the Instruction Sheet.
5. Attach the Nebulizer to the components included in this kit and then connect the supply tube to the gas source. Rotate the Threaded Nut (if included) 1/4 turn to ensure proper fit.
6. Ensure that all connections are secure, and then start the gas flow.
7. Confirm appropriate aerosol function.
8. Discard after single patient use.



MT Promedt Consulting GmbH
Altenhofstr. 80
66386 St. Ingbert, Germany

Westmed

5580 S. Nogales Highway
Tucson, Arizona 85706 USA
Phone: 800-975-7987
Fax: 520-294-6061
www.westmedinc.com
Product of Mexico
Assembled in Mexico
LABEL PN 79305, Rev. 01



LOT