

REF 0365

UNITS - 1
RX Only



NON-REBREATHER MASKS

MASCARILLAS NO REINHALADORAS
MÁSCARAS SEM RECIRCULAÇÃO
MASQUES SANS RÉINSPIRATION
MASCHERE SENZA REINSPIRAZIONE
MASKEN OHNE RÜCKATMUNG
ADULT, ADULTOS, ADULTO, ADULTE,
ADULTI, ERWACHSENE

CAUTION: Use of sterilization or cleaning solutions may leave this device nonfunctional.

SUGGESTED PROCEDURE:

1. Attach the supply tubing to the gas source.
2. Set the oxygen to the desired flow.
3. Check for flow through the device. Always verify that all of the valves function properly:
The valve between the mask and the reservoir should rise on inspiration and lower on exhalation. The valve(s) located on external mask surface should open during exhalation.
4. **Always check oxygen flow through inlet valve before placing the mask on the patient.** Place mask on patient's face. Gently pull each end of the strap until the mask is secure.

Westmed

5580 S. Nogales Highway
Tucson, Arizona 85706 USA
Phone: 800-975-7987
Fax: 520-294-6061

www.westmedinc.com

Assembled in Mexico
LABEL PN 79337, Rev. 01



MT Promedt Consulting GmbH
Altenhofstr. 80
66386 St. Ingbert, Germany
Patent Pending



LOT