5580 S. Nogales Highway • Tucson, AZ 85706 USA

Assembled in Mexico

3105-91VT - 1cc, 25g X 5/8"

3302-91VT - 3cc, 22g X 1"

(English)

(If detached, securely attach needle to syringe) (All syringes contain balanced Lithium/Zinc heparin) (Bovine)

⚠CAUTION • Do not attempt to sterilize or wash this device. • For blood collection only, not for injection.

• Dispose of the needles per internal procedure or in a sharp's disposal container.

Intended Use:

This product is intended for use on the order of a physician as a single use, disposable, syringe to obtain arterial blood samples for in vitro diagnostic use only, for pH blood gas and selected whole blood electrolytes. Do not use for injection. Contents are sterile in unopened, undamaged package.

INSTRUCTIONS FOR USE FOR VENTED SYRINGES

Pre-set Method:

1. Prepare puncture site.
2. Push plunger to bottom of barrel, then set plunger to desired blood sample size.
3. Remove protective sheath from needle by pulling straight off, using caution not to damage needle tip. If using the Q-Cork needle protection system, insert the needle sheath into the hollow end of the Q-Cork stopper. Upon completion of collecting the blood sample, insert the needle (while attached the syringe) into the sheath. Ensure the needle is properly seated into the sheath. Dispose of sheathed needle with Q-Cork into sharps container.
4. Perform arterial puncture in usual manner holding syringe at approximately a °45 angle. Blood will flow into the syringe, venting air out through an internal filter. Note: After the blood has reached the filter, the plunger will pump back by arterial pressure.
5. After obtaining sample, withdraw needle and apply firm digital pressure to puncture site for five minutes or until bleeding has stopped.
6. Insert needle into stopper. With the needle pointed downward, apply light pressure to the plunger. This expels any remaining air and seals the internal filter. If air is not expelled quickly, the filter will seal shortly after becoming wet with blood.
7. Remove needle and attach luer tip cap. Discard needle appropriately in sharps container.
8. Roll syringe between palms for 20 to 30 seconds to thoroughly ix sample.
9. Transport blood sample immediately to laboratory for analysis.

Aspirating method:

1. Prepare puncture site.
2. Hold syringe with needle pointing down and push plunger to bottom of barrel.
3. Remove protective sheath from needle by pulling straight off, using caution not to damage needle tip.
4. Perform arterial puncture in usual manner holding syringe at approximately a °45 angle.
5. After the blood enters the syringe barrel and wets the filter, aspirate the sample to the desired volume of blood.
6. Follow steps 5 through 9 above (Pre-set Method).

INSTRUCTIONS FOR USE WITH VENTED LUER TIP CAP

1. After desired sample volume has been obtained, hold syringe with luer tip in an up position.
2. Securely attach venting luer tip cap.
3. Carefully push the plunger forward to purge any air bubbles out of the syringe through the venting luer tip cap.
4. Venting luer tip cap will no longer vent when the blood wets inside surface of cap filter.
5. Agitate the syringe for 30 seconds to ensure heparin mixing. Transport to the lab for immediate analysis.

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