
Strapette™

5580 S. NOGALES HIGHWAY TUCSON, AZ 85706 USA
www.myAirLife.com

(English)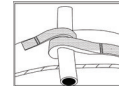

(Italian)

DEVICE DESCRIPTION: Support, Breathing Tube. Non-sterile, disposable, and for single-patient use. to support and anchor a patient's breathing tube.

INTENDED USE: Strapette with Tube Tackles is intended to support and anchor a patient's breathing tube.

INDICATIONS: Indicated for use to support and stabilize breathing tubes.

ENVIRONMENT: Hospitals, sub-acute facilities, medical clinics, pre-hospital, surgical centers, skilled nursing facilities, transport, home.

PATIENT TARGET GROUP: Infants, pediatric

EXPECTED CLINICAL BENEFITS: Functions as a device support and hold breathing tubes for comfort and safety. Helps avoid potential skin irritation.

CONTRAINDICATIONS: Contraindicated in individuals that should not wear a headband because of wounds or malformations of the cranium.

⚠️ WARNINGS:

• Discontinue use if any visible skin irritation occurs. Head straps can cause cranium molding when applied too tight.

⚠️ CAUTIONS:

• Do not attempt to sterilize, autoclave or wash this device.

INSTRUCTIONS: Directions for Use:

1. Remove device from package.
2. Wrap the strap around patient's head with the foam side on the skin.
3. Position the Tube Tackles on the strap under the tubes being secured.
4. Position the tube over the center of the tube tackle and wrap the arms over the tube and attach them to the strap using the hook tabs on each end. Adjust the tightness as needed.

NOTE:

- Small Tube Tackles are sized for high flow oxygen tubes.
- Large Tube Tackles are sized for 10mm CPAP tubes.

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MT Promed Consulting GmbH
MT Promed Consulting Ltd.
MedEnvy UK Limited