(English)

DEVICE DESCRIPTION: Adhesive Free Chin Support. Made with soft skin friendly breathable foam lined material

- Breathable and wicks perspiration
- Compatible with most nasal cannula
- Available in two sizes"

Non-sterile, disposable, and for single-patient use. It functions as a device to support the patient's chin in a closed mouth position to facilitate nasal breathing.

INTENDED USE: Intended to support the chin so the mouth stays closed, thereby causing the patient to breathe through their nose.

INDICATIONS: Indicated for use to promote nasal breathing.

ENVIRONMENT: Hospitals, sub-acute facilities, medical clinics, pre-hospital, surgical centers, skilled nursing facilities, transport.

PATIENT TARGET GROUP: Infants, pediatric

EXPECTED CLINICAL BENEFITS: The device supports the chin so the mouth stays closed, thereby causing the patient to breathe through their nose. Reduce the chance of the patient breathing through the mouth.

CONTRAINDICATIONS: None known.

WARNING:

- Discontinue use if any visible skin irritation occurs.

CAUTIONS:

- Replace if material becomes worn or dirty
- For use with Patients under professional supervision
- Discontinue immediately if skin irritation occurs
- To be used by or on the order of a physician"

INSTRUCTIONS: Directions for Use:

- Remove device from package.
- 1. Place the strap with the slit under the chin.
- 2. Position the curved strap over the top of the head.
- 3. Adjust the hook tabs to hold the chin up.
- 4. Adjust strap to allow the baby to open mouth as needed.

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