Septal-H™ Westmed

5580 S. NOGALES HIGHWAY TUCSON, AZ 85706 USA
(800) 975-7987 www.westmedinc.com

(English)

DEVICE DESCRIPTION: Hydrocolloid material used to protect fragile skin and support nasal breathing tubes. Non-sterile, disposable, and for single-patient use. It functions as a device for skin protection and tube support.

INTENDED USE: The Septal H is intended to be used to prevent injury to the nasal septum of newborn infants when they are fitted with a nasal cannula.

INDICATIONS: Indicated for use to protect newborn skin during treatment with a nasal breathing tube.

ENVIRONMENT: Hospitals, sub-acute facilities, medical clinics, pre-hospital, surgical centers, skilled nursing facilities, transport.

PATIENT TARGET GROUP: Infants, pediatric

EXPECTED CLINICAL BENEFITS: Functions as a device protect the skin and support nasal cannula in place. Aids patient comfort and reduce potential skin irritation.

CONTRAINDICATIONS: None known.

WARNINGS:

• Discontinue use if any visible skin irritation occurs.

CAUTIONS:

• Not for use to stabilize ET Tubes.

INSTRUCTIONS: Directions for Use:

- Remove device from package.
- (Optional) Apply the Hook Tab to the larger side of Septal H and the Loop Tab to the underside of the Nasal Cannula.
- Clean the nasal area and dry the skin
- Apply the larger part of Septal-H to the upper lip
- Hold it down a few seconds as the material responds to the warmth of your hands and the patient's skin.
- The cannula device may now be applied.

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MT Promedt Consulting GmbH
Made in U.S.A.