**(English)**

**DEVICE DESCRIPTION:** Support and anchor oxygen delivery tubes. Non-sterile, disposable, and for single-patient use.  
**INTENDED USE:** The device is intended to be used to support oxygen cannula tubing to avoid contact with the patient's ear.  
**INDICATIONS:** The Head Strap with Tube Tackles is used to keep the tubes in a comfortable position when a patient is receiving oxygen.  
**ENVIRONMENT:** Hospitals, sub-acute facilities, medical clinics, pre-hospital, surgical centers, skilled nursing facilities, transport, home.  
**PATIENT TARGET GROUP:** Adult, pediatric  
**EXPECTED CLINICAL BENEFITS:** Functions as a device support and hold oxygen tubes for comfort and safety. Helps avoid potential skin irritation.  
**CONTRAINDICATIONS:** None known.  
**WARNINGS:**  
• Head bands may be uncomfortable if wrapped too tight.  
**CAUTIONS:**  
• Do not attempt to sterilize, autoclave or wash this device.

**INSTRUCTIONS:** Directions for Use:

1. Remove device from package.
2. Wrap the strap around the head as shown to the left.
3. Position the tube tackles above each ear.
4. Wrap the tube tackles around the cannula tubing to keep it off the ear.

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MT Promedt Consulting GmbH
Made in U.S.A.