
EtCO2 Nasal Sampling Cannula

REF 4000, 4007, 4008, 4100, 4101, 4102, 4103, 4108, 4200, 4208, 4300 and Associated SO PNs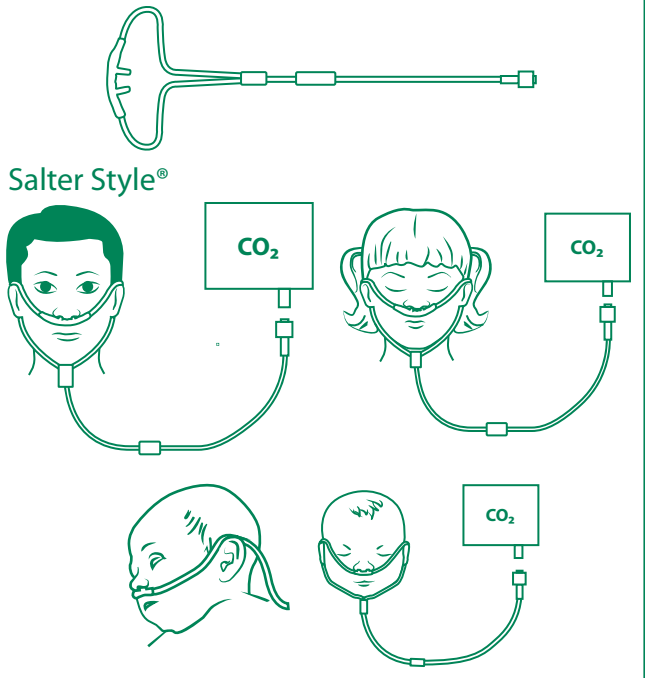

EN EtCO2 Nasal Sampling Cannula

DEVICE DESCRIPTION: The device is a non-sterile, disposable, single patient use cannula that allows sampling of patients exhaled gases.

INTENDED PURPOSE: The Nasal EtCO2 Sampling Cannula is intended to be used to sample exhaled gas.

INDICATIONS FOR USE: The Nasal EtCO2 Sampling Cannula is intended to be used to sample exhaled gas in spontaneously breathing patients requiring expired gas monitoring.

ENVIRONMENT: Hospital, sub-acute, pre-hospital.

PATIENT TARGET GROUP: Infant, pediatric, adult.

EXPECTED CLINICAL BENEFITS: Capnography provides early detection and warning of changes in a patient’s ventilatory status and cardiac perfusion. This enables to clinicians to adjust medication dosage or apply therapeutic intervention to prevent hypoventilation.

CONTRAINDICATIONS: None Known.

WARNINGS:

- If oxygen is in use, do not use near flame or heat source.
- Position tubing to avoid strangulation.
- Patient may become hypoxic if oxygen flow is interrupted.
- Keep excess tubing loosely coiled and out of the way to prevent kinking and tripping hazard.
- To reduce the risk of misconnections and patient injury, always trace tubing from gas source to the medical device before connecting.

- Contains Tris(nonylphenyl) phosphite (TNPP) above 0.1% w/w.

CAUTIONS:

- Consult physician if patient develops infection, skin irritation or material sensitization.
- Do not place anything on supply tubing that may obstruct flow.
- Not for use on patients with obstruction of both nostrils.
- Excessive patient secretion or a build-up of liquids in the tubing may occlude the sampling and require more frequent replacement.

RESIDUAL RISKS: Refer to Warnings and Cautions.

NECESSARY USER QUALIFICATIONS: This device is restricted to sale by or on the order of a physician. There are no additional requirements for special facilities, special training, or particular qualifications for the use of this device.

INSTRUCTIONS: Follow instructions to avoid risk of infection and contamination.

- Connect the gas sampling line to the capnography monitor.
- Insert prongs into nares.
- Wrap headset loop up and over ears.
- Squeeze the sides of the bolo and glide up under the chin.
- Check monitor display for EtCO2 value.

This product is disposable and is not intended for prolonged use greater than 7 days.

SAFE DISPOSAL: Dispose of device in accordance with local, state or national regulations.

INCIDENT REPORTING: Contact your country’s Competent Authority and Manufacturer to report any serious incident.

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