Valved Aerosol Mask

EN Valved Aerosol Mask

DEVICE DESCRIPTION: The device is non-sterile, disposable, and for single-patient use. The aerosol valved mask includes one-way valves placed over the exhalation hole to aid in medication delivery.

INTENDED PURPOSE: Aerosol masks are intended to be used for aerosolized medication delivery applications.

INDICATIONS: Aerosol mask is used in conjunction with a nebulizer to deliver aerosolized medication to the mouth and nose in spontaneously breathing patients.

ENVIRONMENT: Hospitals, sub-acute, medical clinics, pre-hospital, home, surgical centers, skilled nursing facilities.

PATIENT TARGET GROUP: Pediatric, adult.

EXPECTED CLINICAL BENEFITS: When used to deliver bronchodilators, clinical benefit is to relieve bronchospasms, decrease the frequency of exacerbations and improve lung function.

CONTRAINDICATIONS: None Known.

WARNINGS:

- If oxygen is in use, do not use near flame or heat source.
- Device contains components that may present choking hazard.
- Use a flow rate ≥ 5 LPM to prevent CO₂ accumulation and re-breathing.
- Contains Octamethylcyclotetrasiloxane (D4) above 0.1% w/w.

CAUTIONS:

- Clinician or caregiver should avoid inhaling aerosolized medication.
- Consult physician if patient develops infection, skin irritation, or material sensitization.

RESIDUAL RISKS: Refer to Warnings and Cautions.

NECESSARY USER QUALIFICATIONS: This device is restricted to sale by or on the order of a physician. There are no additional requirements for special facilities, special training, or particular qualifications for the use of this device.

INSTRUCTIONS: Follow instructions for proper care of device to prevent risk of contamination and injury.

- Place mask over nose and mouth.
- Adjust elastic strap and nose bar for snug fit.
- Insert connector into nebulizer or connect to aerosol tubing.

4. Set prescribed flow rate.

5. Verify aerosol mist is visible.

This product is disposable and is not intended for prolonged use greater than seven days.

SAFE DISPOSAL: Dispose of device in accordance with local, state or national regulations.

INCIDENT REPORTING: Contact your country's Competent Authority and manufacturer to report any serious incident.

PERFORMANCE CHARACTERISTICS:

- For use with Flow Rate 5 - 8 LPM
- For use with 22 mm connector

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