## Skin Fixation Tender Grips

|   |
|---|
| <b>EN</b>   |
| <b>DEVICE DESCRIPTION:</b> The device is non-sterile, disposable and for single-patient use. Tender Grip® is an adhesive device used to secure cannula or tubing to patient's skin. Tender Grip is a circular adhesive tape with a fixation strap to secure tubing. <p><b>INTENDED PURPOSE:</b> Tender Grips are a fixation device used to hold the nasal cannula tubing in place.</p> <p><b>INDICATIONS:</b> Skin fixation accessories to provide a means of affixing cannula or tubing to the face and body.</p> <p><b>ENVIRONMENT:</b> Hospital, out-patient, trasport, home, extended care</p> <p><b>PATIENT TARGET GROUP:</b> Newborns, infant, pediatric, adult</p> <p><b>EXPECTED CLINICAL BENEFITS:</b> Maintains cannula position to help prevent interruption of therapy.</p> <p><b>CONTRAINDICATIONS:</b> Not for use with endotracheal tubes. Not for use over broken skin or on patients with known sensitivity to device materials</p> <p><b>WARNINGS:</b> Do not secure to tracheostomy tube with tape or any method other than friction fit. Device contains components that may present choking hazard.</p> <p><b>CAUTIONS:</b> Consult physician if patient develops infection, skin irritation or material sensitization. Follow cleaning instructions to avoid risk of infection and contamination.</p> <p><b>INSTRUCTIONS:</b> Contact your country’s Competent Authority to report any serious incident. Dispose of device in accordance with local, state or national regulations. Follow instructions for proper care of device to prevent risk of contamination and injury. Gently remove to prevent skin damage. Replace if device becomes loose and starts to lift away from skin.</p> |

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MT Promedt Consulting GmbH
MT Promedt Consulting Ltd.
MedEnvoy UK Limited