Oxygen Mask Venturi

EN Oxygen Mask Venturi

DEVICE DESCRIPTION: The device is non-sterile, disposable and for single-patient use.

• ***Percent O2 Lock:*** A single unit variable venturi system designed to deliver seven fixed oxygen concentrations via an adult mask to a spontaneously breathing patient. The concentration of oxygen range from 24 to 50%.

• ***Accu-Flow:*** A single unit interchangeable venturi system designed to deliver five fixed oxygen concentration via an adult mask to a spontaneously breathing patient. The concentration of oxgen range from 24 to 40%.

INTENDED PURPOSE: A venturi mask is a device containing an air-oxygen mixing mechanism that dilutes 100% oxygen to a predetermined concentration and delivers the mixed gases to a patient.

INDICATIONS FOR USE: For patient's requiring supplemental oxygen. To treat hypoxia, relieve shortness of breath and pre-oxygen-ate for surgical procedures in spontaneously breathing patients.

ENVIRONMENT: Hospitals, sub-acute, medical clinics, pre-hospital, home, surgical centers, skilled nursing facilities.

PATIENT TARGET GROUP: Pediatric, adult.

EXPECTED CLINICAL BENEFITS: The oxygen mask delivers up to 8 LPM of oxygen to reverse or prevent hypoxemia, and provide relief of shortness of breath in oral and nasal breathers.

CONTRAINDICATIONS: None Known.

WARNINGS:

- If oxygen is in use, do not use near flame or heat source.
- Position tubing to avoid strangulation.
- Device contains components that may present choking hazard.
- Patient may become hypoxic if oxygen flow is interrupted.
- Not intended to be used with oxygen concentrator.
- Keep excess tubing loosely coiled and out of the way to prevent kinking and tripping hazard.
- To reduce the risk of misconnections and patient injury, always trace tubing from gas source to the medical device before connecting.

CAUTIONS:

- Consult physician if patient develops infection, skin irritation or material sensitization.
- Do not place anything on supply tubing that may obstruct flow.

RESIDUAL RISKS: Refer to Warnings and Cautions.

NECESSARY USER QUALIFICATIONS: This device is restricted to sale by or on the order of a physician. There are no additional require-ments for special facilities, special training, or particular qualifications for the use of this device.

INSTRUCTIONS: Follow instructions to avoid risk of infection and contamination.

Percent O2 Lock:

- To set delivered oxygen %, pull the sleeve back until the spring is completely compressed, then turn the sleeve to the prescribed setting.
- Release the spring to lock sleeve in place.
- Set the recommended flow rate.
- Place mask over nose and mouth.
- Adjust elastic strap and nose bar for a snug fit.
- Check for leaks at connection and check the medical device works properly.
- To add humidity, slip the humidifier cup over the venturi/diluter jet and connector aerosol tubing to humidifier cup.

Accu-Flow:

- Select the diluter jet for the desired FiO2.
- Attach to corrugated tube on mask.
- Set the recommended flow rate.
- Place mask over nose and mouth.
- Adjust elastic strap and nose bar for a snug fit.
- Check for leaks at connection and check the medical device works properly.
- To add humidity, slip the humidifier cup over the venturi/diluter jet and connector aerosol tubing to humidifier cup.

This product is disposable and is not intended for prolonged use greater than 14 days.

SAFE DISPOSAL: Dispose of device in accordance with local, state or national regulations.

INCIDENT REPORTING: Contact your country's Competent Authority and Manufacturer to report any serious incident.

PERFORMANCE CHARACTERISTICS:

• For use with Flow Rate 3 - 8 LPM

• For use with Bonded O2 Tubing adaptor

The unit should provide the following percentage of oxygen at these stated flows:

Percent O2 Lock		Accu-Flow	
3 LPM:	24 ± 1%	6 LPM:	35 ± 2%
	26 ± 1%		40 ± 2%
	28 ± 1%		50 ± 2%
	31 ± 1%		

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MT Promedt Consulting GmbH