REF 2D0735X/2D0737

AirLife™ Sterile Water for Inhalation, USP, 1000 ml and 2000 ml

Nonpyrogenic pH ~ 5 • Applicable for Canada only: Medicinal Ingredient: Water 1 g/mL • **Recommended storage:** Room temperature (25°C or 77°F). Avoid excessive heat. Protect from freezing.

- REF** Catalogue number U.S. Federal law restricts this device to sales by or on the order of a physician
- LOT** Lot Number Manufacturer
- QTY** Lot Number Use by Date
- PHT DEHP** Contains Phthalates Do not use if package is damaged
- Consult instructions for use For Single Use Only
- Sterilize Fluid path Dry Heat Keep away from sunlight
- Caution

en

English

Caution:

- Squeeze and inspect inner bag which maintains product sterility.
- Discard if leaks are found.
- For inhalation therapy only. Not for parenteral administration.
- Do not use unless solution is clear and container is undamaged.
- Discard unused contents of opened containers.
- Store unit in moisture barrier overpouch at room temperature until ready to use.
- For single use only. Re-use may degrade the performance of the product or contribute to cross contamination.

USA Federal (USA) law restricts this device to sale by or on the order of a physician or licensed practitioner. Dosage and administration as directed by physician.

Instructions for use: Use aseptic technique.

1. Open overpouch, tear down from slit.
2. Suspend container from eyelet support.
3. Remove outside half of twist-off connector.
4. Close control clamp of transfer set and insert spike well into the solution port.
5. Connect other end of transfer set to desired respiratory device.
6. Proceed according to respiratory transfer set and device manufacturer instructions.

Contains Phthalates (DEHP). The benefit of treatment with medical respiratory support devices outweighs the remote possibility of exposure to phthalates.

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Cardinal Health Canada, Inc.