ΔVΔNOS*

BALLARD*
MULTI-ACCESS
CATHETER

Instructions for Use

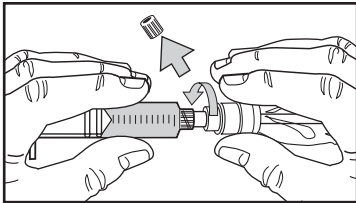
Fig. 1

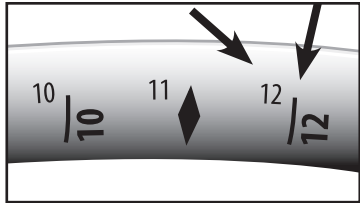
Fig. 2

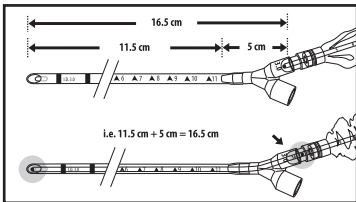
Fig. 3

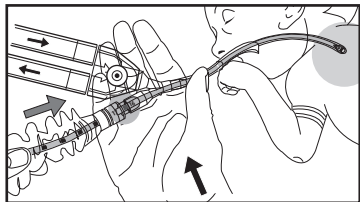
Fig. 4



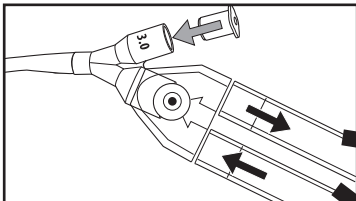
Fig. 5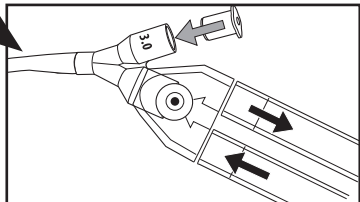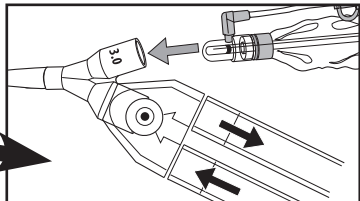

V_D Internal Volume	Diameter	Length	Single Use Only	STERILE R	Do not use if package damaged	Do not resterilize
Not made with natural rubber latex	Product is not made with DEHP as a plasticizer	Rx Only	Caution	Consult instructions for use	Endotracheal Length	

Instructions for Use

Rx Only: Federal (USA) law restricts this device to sale by or on the order of a physician.

Intended Use

The BALLARD* Multi-Access Catheter is intended for use for surfactant delivery to the distal airway.

Warning

- **Do not reuse, reprocess, or sterilize this medical device. Reuse, reprocessing, or sterilization may 1) adversely affect the known biocompatibility characteristics of the device, 2) compromise the structural integrity of the device, 3) lead to the device not performing as intended, or 4) create a risk of contamination and cause the transmission of infectious diseases resulting in patient injury, illness, or death.**
- **Do not trim or cut the endotracheal tube (not supplied) while the BALLARD* Multi-Access Catheter is attached, otherwise the catheter may also be cut and that portion of the catheter may be aspirated into the lower respiratory tract of the patient and may cause death or serious injury.**

Caution

- Single patient use. Do not sterilize.
- Do not use for more than 24 hours.
- This device may change deadspace. Clinical assessment should be made prior to installation.

Setup

1. Attach desired accessory (syringe, sampling or pressure line, etc.) to luer fitting on the distal end of the BALLARD* Multi-Access Catheter. (Fig. 1)
2. Attach the BALLARD* Multi-Access Catheter to the Y-Adapter that corresponds to the size of the endotracheal tube.
3. Attach the Y-Adapter to the patient's endotracheal tube and ventilator circuit.

Use

1. There are two methods for determining proper depth of insertion for the BALLARD* Multi-Access Catheter:
 - **Method 1 (Fig. 2)**
 1. Align any printed depth number on the catheter with the same number printed on the endotracheal tube.
 2. Catheter tip will be within 0.5 to 1 cm of the end of the endotracheal tube.

- **Method 2 (Fig. 3)**

1. Observe the printed depth number on the endotracheal tube closest to the end of the adapter.
 2. Add 5 to this number.
 3. Advance catheter until the cm (depth number plus 5) appears in the area directly distal to the Y-Adapter. Note the nearest color band. It will allow easier visualization on subsequent insertions.
 4. Catheter tip will be within 0.5 to 1 cm of the end of the endotracheal tube.
 5. Color bands allow easier visualization on subsequent insertions.
2. Stabilize the BALLARD* Multi-Access Catheter and Y-Adapter with one hand, then advance the catheter into the endotracheal tube with the thumb and forefinger of the opposite hand. (Fig. 4)
 3. Release catheter and repeat until desired depth is reached.
 4. Measure pressures, flows according to manufacturer recommendations.
- OR**
5. Sample gasses or fluids through the catheter.
- OR**
6. When administering fluids, prior to insertion, fill the internal volume of the BALLARD* Multi-Access Catheter with the fluid that will be used. Insert catheter to desired depth. Administer the appropriate dosage of fluid. Retract the catheter from the endotracheal tube. Allow patient parameters (SpO₂, HR, etc.) to return to baseline.
 7. Repeat until desired dosage is administered.
 8. Remove catheter from Y-Adapter.
 9. Replace the BALLARD* Multi-Access Catheter with the BALLARD* Neonatal Y Closed Suction Catheter or with the 7.5 mm plug provided in the kit. (Fig. 5)

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*Registered Trademark or Trademark of Avanos Medical, Inc., or its affiliates.
© 2018 AVNS. All rights reserved. 2019-07-01 15-M1-181-01 / 70210617
Distributed in the USA by Avanos Medical Sales, LLC,
Avanos Medical, Inc., 5405 Windward Parkway, Alpharetta, GA 30004 USA.
Sponsored in Australia by Avanos Medical Australia Pty Ltd, 475 Victoria Avenue, Chatswood, NSW 2067, Australia.