<span><span></span></span>	<b>AVANOS® Flex Connector</b> <b>Instructions For Use</b>
----------------------------	--

Rx Only: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

- Connect between the endotracheal or tracheostomy tube and either the AVANOS® closed suction catheter or the breathing circuit.
- Adjust for length and position.

⚠**Warning:**

**Do not reuse, reprocess, or resterilize this medical device. Reuse, reprocessing, or resterilization may 1) adversely affect the known biocompatibility characteristics of the device, 2) compromise the structural integrity of the device, 3) lead to the device not performing as intended, or 4) create a risk of contamination and cause the transmission of infectious diseases resulting in patient injury, illness, or death.**

⚠**Caution:**

- Do not over expand the tubing. This could cause loss of pressure or volume.
- Do not soak, rinse, wash or resterilize this product.

<span><span></span></span>	Single Use Only	<span><span></span></span>	STERILE R Sterilized by Irradiation	<span><span></span></span>	Do not use if package damaged	
<span><span></span></span>	Do not resterilize	Rx Only	<span><span></span></span>	Caution	<span><span></span></span>	Consult instructions for use

<span><span></span></span>	<span><span></span></span>	<span><span></span></span>	<span><span></span></span>
----------------------------	----------------------------	----------------------------	----------------------------

<span><span></span></span>	<span><span></span></span>	<span><span></span></span>	<span><span></span></span>
----------------------------	----------------------------	----------------------------	----------------------------

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