


AVANOS*

BALLARD* SUCTION SWAB PACK

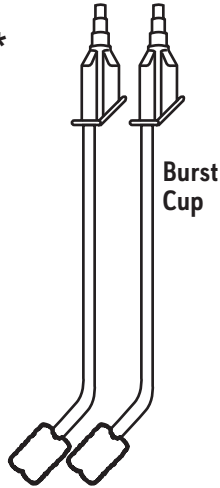


- Contents:
- 2 Angled Suction Swabs with Thumb Port
 - 1 0.5 fl. oz. 1.5% Mint Flavored Hydrogen Peroxide

Caution:
Follow universal precaution guidelines. Exercise caution in patients with altered levels of consciousness.

WARNING: KEEP OUT OF REACH OF CHILDREN!
Avoid contact with eyes. If contact occurs, flush thoroughly with cool water for at least 15 minutes.

	Single Use Only	Store at room temperature	REF 33971
--	-----------------	---------------------------	------------------

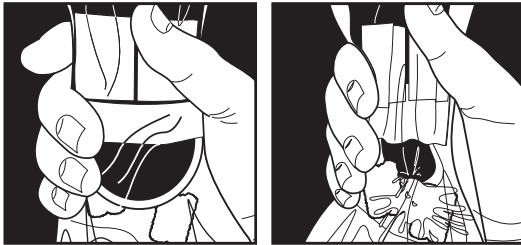

Squeeze pack as indicated to burst cup, to moisten swabs before opening.

Oral Debriding Agent

Drug Facts

Active Ingredient	Purpose
Hydrogen Peroxide 1.5%	Oral Debriding Agent

Uses

- Aids in the removal of phlegm, mucus, or other secretions in the temporary relief of discomfort due to occasional sore throat and sore mouth.

Warnings

Stop use and ask a doctor if:

- Swelling, rash, or fever develop.
- Severe or persistent sore throat or sore throat accompanied by high fever, headache, nausea, and vomiting occurs.
- Do not use more than 2 days or administer to children under 3 years of age unless directed by a physician.

Keep out of reach of children under 3 years of age.

Directions

- Topical dosage for adults and children 3 years of age and older is a rinse used no more than 4 times daily. For children under 3 years of age, there is no recommended dosage except under the advice and supervision of a dentist or doctor.
- Use only under health care practitioners supervision.

Other Information

- Store at room temperature. Avoid excessive heat. Avoid freezing.

Inactive Ingredients

Purified Water, Glycerin, Flavor, Sodium Saccharin

Questions or Comments?

1-844-428-2667

Distributed in the USA by Avanos Medical Sales, LLC, 5405 Windward Parkway, Alpharetta, GA 30004 USA. In USA, 1-844-428-2667. www.avanos.com
 Avanos Medical, Inc., 5405 Windward Parkway, Alpharetta, GA 30004 USA.
 EC [REF] Avanos Medical Belgium BVBA, Leonardo Da Vincilaan 1, 1930 Zaventem, Belgium. Sponsored in Australia by Avanos Medical Australia Pty Ltd, 475 Victoria Avenue, Chatswood, NSW 2067, Australia.
 *Registered Trademark or Trademark of Avanos Medical, Inc., or its affiliates.
 © 2018 AVNS. All rights reserved.

06-M1-162-01 / 50001232

Product of Mexico

LOT

Use By

