INSTRUCTIONS FOR USE

- Select appropriate CPAP Cannulaide device size according to infant's weight.
- Before application, clean infant's face with sterile water and dry completely.
- To improve adhesion, warm CPAP Cannulaide device between hands or under bed warmer.
- Gently peel CPAP Cannulaide device from the sheet.
- Hold the CPAP Cannulaide device by the ends of the mustache strip, and align the nasal openings on the nose petals with infant's nares.
- Apply the mustache strip above the upper lip with adhesive side down. Fold the petals up and around the nose. Gently press down and smooth CPAP Cannulaide.

7a. Across-the-Face Style: Wrap the sticky loop tape around each side of the CPAP cannula barrel. Insert prongs through CPAP Cannulaide nasal openings. Attach CPAP cannula to hook strip on mustache and secure CPAP interface to headpiece. Ensure there is a small space between septum and cannula.

7b. Over-the-Head Style: Insert nasal prongs through the CPAP Cannulaide nasal opening. TIP: A thin layer of water soluble lubricant on prongs may aid in insertion. Attach CPAP interface to nasal prongs and secure to head piece. Ensure the nasal interface is not pressed against the septum.

8. Monitoring and Routine Care:

- Connect to CPAP delivery device and ensure prescribed pressure is being delivered.


- Inspect for proper placement of CPAP Cannulaide securement device and skin integrity at least every 2 hours.

- Routinely inspect the insides of the nares and provide gentle suction as needed to remove moisture and secretions.

- Replace CPAP Cannulaide device at least once every 24 hours or more frequently if needed.

- Follow your facility's protocol for management and monitoring of CPAP therapy and equipment.

- Update your nasal CPAP protocol to include the proper use of the CPAP Cannulaide.

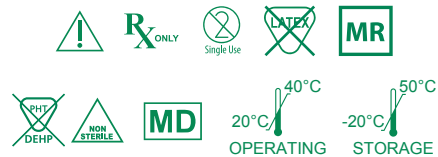DESCRIPTION:

CPAP Cannulaide has a hydrocolloid backing which is applied above the lip and over the nose. It is an accessory to infant nasal CPAP interface, and not intended to replace the primary securement device.

INDICATIONS FOR USE:

To aid in securing and positioning the nasal interface for neonates and infants undergoing non-invasive ventilation in an acute care setting. CPAP Cannulaide is a disposable device and is for single patient use only.

WARNINGS:

- Do not use with low or high flow oxygen nasal cannulas.
- Do not use to secure endotracheal tubes.
- Excess condensation in the CPAP circuit and interface may interfere with the CPAP Cannulaide adhesion properties. Follow your facility's procedure for managing circuit condensation.
- Do not use with a nasal CPAP masks.

CAUTIONS:

- Do not use if known sensitivity to hydrocolloids.
- If skin irritation or necrosis is noted, discontinue use or do not initiate use of the CPAP Cannulaide device.
- If systemic or local infection develops under the CPAP Cannulaide device, discontinue use and initiate appropriate adjunctive therapy.
- Remove CPAP Cannulaide if CPAP is discontinued or if nasal prongs are removed.

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				
Salter Labs® 30 Spur Drive El Paso, TX 79906 USA	MT Promedet Consulting GmbH Ernst-Heckel-Strasse 7 66386 St. Ingbert Germany	MT Promedet Consulting Ltd. Beaver House, 23-38 Hythe Bridge Street Oxford, OX1 2EP United Kingdom	MedEnvoy Switzerland Gotthardstrasse 28 6302 Zug Switzerland	
				
www.MyAirLife.com Made in USA Packaged in Mexico	MedEnvoy NL-IM-000000248 Prinses Margrietplantsoen 33 Suite 123 2595 AM, The Hague The Netherlands	MedEnvoy UK Limited 85, Great Portland Street – First Floor London, W1W 7LT United Kingdom		