EN

Ethox® Combiport® IV bag spike and Combiguard™ Shield directions for use


CAUTION:
Use standard hospital procedure for wound irrigation, gastric lavage and IV infusion. Use universal precautions and wear protective equipment.

Combiport® use for wound irrigation

1. Use IV saline bag containing volume of fluid recommended by physician for wound irrigation.
2. Spike IV saline solution bag with Combiport®.
3. Remove cap and squeeze bag to create irrigation stream.
4. A Combiguard™ anti-splash device may be used. See instructions below.
5. For higher pressure or thinner irrigation stream during irrigation, securely attach standard IV catheter tip of the Combiport® in place of the Combiguard™.
6. When irrigation is complete, recap Combiport®.
7. Dispose after single use.

Combiguard™ use for anti-splash device during wound irrigation

1. Aseptically attach Combiguard™ to male luer tip of Combiport®, syringe or irrigating system.
2. During irrigation move Combiguard™ in direction of arrow on device.
3. Capture irrigating splash under Combiguard™ and direct drainage with outlet openings.
4. Dispose after single use.

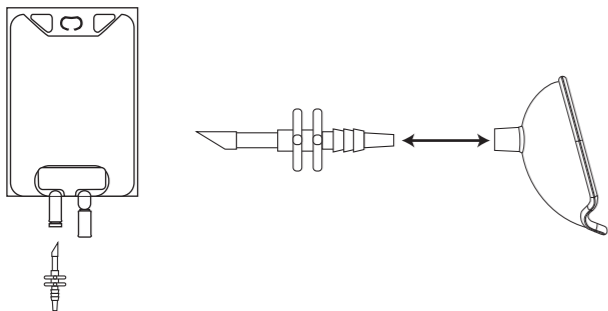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