

CPAP CANNULAIDE® SECUREMENT DEVICE

MONITORING AND ROUTINE CARE

- Connect to the CPAP delivery device and ensure the prescribed pressure is being delivered.
- Inspect skin integrity and for proper placement of the CPAP Cannulaide® every 2 hours.
- Routinely inspect the insides of the nares and provide gentle suction as needed to remove moisture and secretions.
- Replace the CPAP Cannulaide device at least once every 24 hours or more frequently if needed.
- Follow your facility's protocol for management and monitoring of the CPAP therapy.

APPLICATION TIPS:

- Have the CPAP system set up and apply the CPAP fixation device, e.g. bonnet prior to application of the CPAP Cannulaide.
- If using an across-the-face interface, apply the sticky loop to the CPAP cannula prior to applying the Cannulaide to the infant.
- A thin layer of water soluble lubricant or water on the prongs may aid in insertion into the nares.

Consult instruction for use for additional information, warnings and cautions.


1 SELECT SIZE

ITEM	WEIGHT	SIZE
CA100	< 700 g	0
CA101	700g - 1250 g	1
CA102	1251g - 2000 g	2
CA103	2001g to 3000 g	3
CA104	> 3000 g	4


2



3

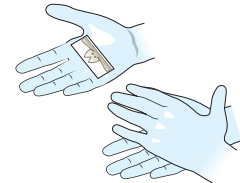

4

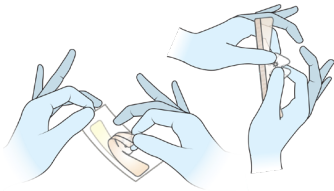


5



6



7A ACROSS-THE-FACE CPAP INTERFACE

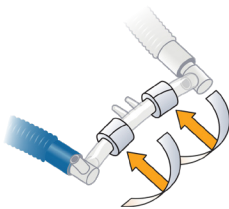

7B OVER-THE-HEAD CPAP INTERFACE

