

# BALLARD™ ASSISTED CARE 24-HOUR ORAL CARE KIT

## STEP BY STEP INSTRUCTION GUIDE\*

### GETTING STARTED

Attach Yankauer to Suction Handle



**REMEMBER**

- Use Yankauer as needed during kit use (24 hours)
- Suction Handle attaches to all components in the kit
- For a secure fit with Suction Handle, **twist components 1/4 turn**
- **Do not throw away Suction Handle or Yankauer** during kit use (24 hrs)

### STEP 1

Morning  
Time: \_\_\_\_\_  
Clinician: \_\_\_\_\_



#### OPEN PURPLE TOOTHBRUSH PACK

- **Connect** Toothbrush to Handle, **dip** in solution cup
- **Brush** teeth 1-2 minutes, and **suction** often
- **Moisturize** lips and mouth with Oral Swab and Mouth Moisturizer
- **Discard** Toothbrush and Swabs, and **reattach** Yankauer to Handle

### STEP 2

Afternoon  
Time: \_\_\_\_\_  
Clinician: \_\_\_\_\_



#### OPEN GREEN SUCTION PACK

- **Connect** Suction Swab to Handle, **dip** in solution cup
- **Swab** mouth, and **suction** often
- **Moisturize** lips and mouth with Oral Swab and Mouth Moisturizer
- **Discard** Swabs, and **reattach** Yankauer to Handle

### STEP 3

Evening  
Time: \_\_\_\_\_  
Clinician: \_\_\_\_\_



#### OPEN PURPLE TOOTHBRUSH PACK

- **Connect** Toothbrush to Handle, **dip** in solution cup
- **Brush** teeth 1-2 minutes, and **suction** often
- **Moisturize** lips and mouth with Oral Swab and Mouth Moisturizer
- **Discard** Toothbrush and Swabs, and **reattach** Yankauer to Handle

### STEP 4

Bedtime  
Time: \_\_\_\_\_  
Clinician: \_\_\_\_\_



#### OPEN BLUE SUCTION SWAB PACK

- **Connect** Suction Swab to Handle, **dip** in solution cup
- **Swab** mouth, and suction often
- **Moisturize** lips and mouth with Oral Swab and Mouth Moisturizer
- **Discard** Swabs, and **reattach** Yankauer to Handle

\*See Order of Use Guide on back panel of Assisted Care kit