


Remember... the gray stays and the green goes!

Bedside Monitoring

Your facility has recently transitioned to the Multi-Link™ X2 ECG System from AirLife®. The **green** leadwires are disposable and travel with the patient. Remember to disconnect the leadwires from the gray cable when transferring the patient. The **gray** ECG cables are reusable and dedicated to the room. They should remain attached to the monitor.



Reusable ECG Cable

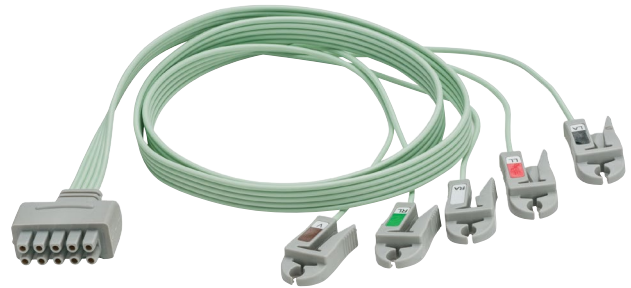


Disposable Leadwire

Telemetry Monitoring

In a telemetry environment the **gray** adapter stays with the telemetry transmitter and the **green** leadwire travels with the patient.



Telemetry Adapters

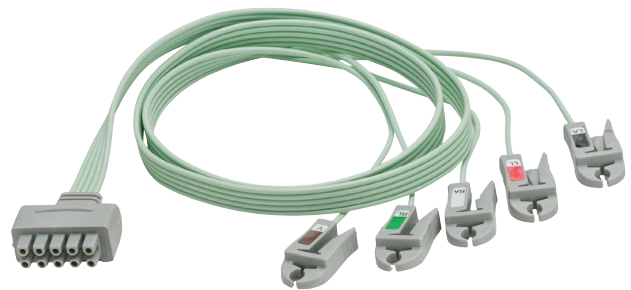

Disposable Leadwire