
# Instructions for Use: Pressure Infusion Bag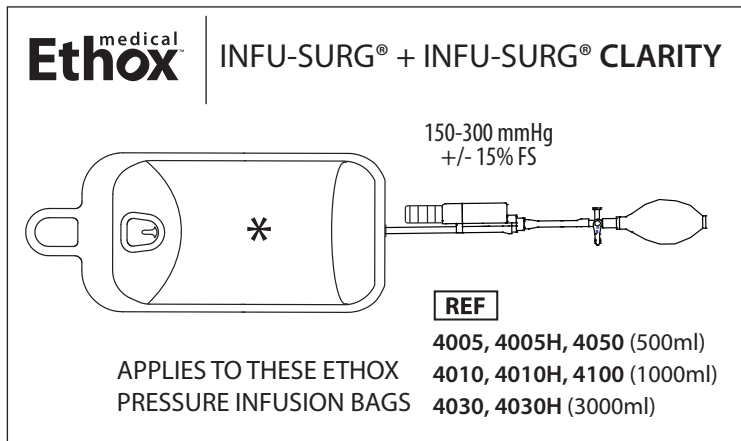

**EN** **Indications for Use:** Single patient use device that is intended to pressurize the IV bags which in turn deliver a variety of fluids/medications to human body.

**Instructions for Use:**

1. Insert intravenous bag between mesh/panel (\*) and pressure bag.
2. Hang intravenous bag on mounted hook or slide hanger loop through top of IV bag if applicable.
3. Inflate: Close valve on inflation bulb. Pump inflation bulb until gauge indicates the proper pressure.
4. Rotate stopcock to "Maintain Pressure" position.
5. Deflate: Open valve on inflation bulb.

**Caution:** For best transfer of pressure to the IV Bag, ensure fluid bag is placed within the pressure infuser panel.

**MRI Safety Information**

The Ethox Pressure Infusion Bag is MR Conditional under the following conditions.

- **Name of the Device:** Ethox Pressure Infusion Bag
- **Static Magnetic Field (T):** 7-T or less
- **Maximum Spatial Field Gradient (T/m and gauss/cm):** 19-T/m (1,900-gauss/cm)

**Warning:** Because the fluid bag pressure will decrease as fluid is evacuated from the fluid bag, monitor the gage pressure continually and inflate as needed.

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