

ΔVΔNOS*



**BALLARD* LIBERATOR
MANIFOLD CAP SET**

Instructions for Use



General Instructions for Use

These instructions apply to: BALLARD* Liberator Manifold Cap Set.

Indications:

The BALLARD* Liberator Manifold Cap Set is a single patient use accessory for use with the BALLARD* Liberator ET Tube Clearing System.

⚠ Warnings








- Do not cut or trim the endotracheal tube (not supplied) while the BALLARD* Liberator ET Tube Clearing System are attached, otherwise the BALLARD* Liberator Closed Suction Catheter may also be cut and a portion of the catheter could be aspirated into the lower respiratory tract and may cause death or serious injury.**
- Do not reuse or reprocess this medical device. Reuse or reprocessing may: (a) adversely affect the known biocompatibility of the device, (b) compromise the structural integrity of the device, (c) lead to the device not performing as intended, (d) create a risk of contamination and cause the transmission of infectious diseases resulting in patient injury, illness or death.**

⚠ Cautions


- Single patient use only.
- Rx only.
- For use only with the BALLARD* Liberator Closed Suction ET Tube Clearing System.
- Do not use if packaging has been compromised. Nonsterile contents may cause infection.
- Refer to BALLARD* Liberator ET Tube Clearing System instructions for use prior to use of the BALLARD* Liberator Manifold Cap Set.

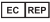
Suggested Procedure

- Refer to BALLARD* Liberator ET Tube Clearing System instructions for use prior to use of the Cap Set.
- Using gloves and eye protection, carefully open the sterile package in a clean manner.
- Assure stopcock on manifold is in the ventilator only position.
- Disengage cap set by separating blue cap (manifold cap) from white cap (catheter cap).
- Carefully disconnect catheter from manifold rotating counter clockwise. Avoid leaving device/accessory disconnected from manifold for prolonged period of time.
- Using white cap, place cap on catheter (clockwise twist). Set catheter aside.
- Carefully connect blue cap to manifold.
- To reconnect catheter, assure stopcock on manifold is in the ventilator only position.
- Carefully disconnect blue cap from manifold.
- Disconnect white cap from catheter.
- Carefully connect catheter to manifold rotating clockwise until positive engagement.
- Mate blue cap to white cap and set aside.

 Single Use Only		 Do not use if package is damaged	 Do not resterilize
 Not made with natural rubber latex	Rx Only	 Caution	 Consult instructions for use

Distributed in the USA by Avanos Medical Sales, LLC,
5405 Windward Parkway, Alpharetta, GA 30004 USA.
In USA, 1-844-4AVANOS (1-844-428-2667). www.avanos.com

 Avanos Medical, Inc.,
5405 Windward Parkway,
Alpharetta, GA 30004 USA.

 Avanos Medical Belgium BVBA,
Leonardo Da Vincilaan 1,
1930 Zaventem, Belgium.

Sponsored in Australia by Avanos Medical
Australia Pty Ltd, 475 Victoria Avenue,
Chatswood, NSW 2067, Australia.

*Registered Trademark or Trademark of Avanos Medical, Inc.,
or its affiliates. © 2018 AVNS. All rights reserved. 2023-08-16
15-M1-839-03