AirLife® Salter Labs® Oxygen Mask Partial Rebreather

REF 13120, 13124

EN Oxygen Mask Partial Rebreather

DEVICE DESCRIPTION: The device is non-sterile, disposable and for single-patient use. A partial rebreather mask is a high concentration oxygen mask placed over a patient's nose or mouth.

INTENDED PURPOSE: An oxygen mask is a device placed over a patient's nose or mouth to administer oxygen.

INDICATIONS FOR USE: For patient's requiring supplemental oxygen. To treat hypoxia, relieve shortness of breath and pre-oxygenate for surgical procedures in spontaneously breathing patients.

ENVIRONMENT: Hospitals, sub-acute, medical clinics, pre-hospital, home, surgical centers, skilled nursing facilities.

PATIENT TARGET GROUP: Pediatric, adult.

EXPECTED CLINICAL BENEFITS: The oxygen mask delivers up to 25 LPM of oxygen to reverse or prevent hypoxemia, and provide relief of shortness of breath in oral and nasal breathers.

CONTRAINDICATIONS: None Known.

WARNINGS:

- If oxygen is in use, do not use near flame or heat source.
- Position tubing to avoid strangulation.
- Device contains components that may present choking hazard.
- Patient may become hypoxic if oxygen flow is interrupted.
- Set flow rate to ≥ 8 LPM to remove exhaled CO₂ and to refill oxygen reservoir.
- Keep excess tubing loosely coiled and out of the way to prevent kinking and tripping hazard.
- To reduce the risk of misconnections and patient injury, always trace tubing from gas source to the medical device before connecting.

CAUTIONS:

- Consult physician if patient develops infection, skin irritation or material sensitization.
- Do not place anything on supply tubing that may obstruct flow.

Residual Risks: Refer to Warnings and Cautions.

NECESSARY USER QUALIFICATIONS: This device is restricted to sale by or on the order of a physician. There are no additional requirements for special facilities, special training, or particular qualifications for the use of this device.

INSTRUCTIONS: Follow instructions to avoid risk of infection and contamination.

- 1) Attach supply to gas source and set flow 8 to 10 LPM.
- 2) Place mask over nose and mouth.
- 3) Adjust elastic strap and nose bar for a snug fit.
- 4) Adjust flow to keep reservoir bag 1/3 to 1/2 inflated.
- 5) Check for leaks at connections and for proper placement prior to use.

This product is disposable and is not intended for prolonged use greater than 14 days.

Safe Disposal: Dispose of device in accordance with local, state or national regulations.

Incident Reporting: Contact your country's Competent Authority and Manufacturer to report any serious incident.

PERFORMANCE CHARACTERISTICS:

- For use with Flow Rate 3 - 25 LPM
- For use with Bonded O₂ Tubing adaptor

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MT Promedt Consulting GmbH
MT Promedt Consulting Ltd.
MedEnvoy UK Limited