AVANOS*

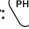
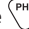
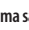
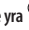
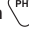
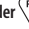
BALLARD*
ORAL CARE
ACCESSORIES

Instructions for Use

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⚠ Warning: Do not reuse or reprocess this medical device. Reuse or reprocessing may 1) adversely affect the known biocompatibility characteristics of the device, 2) compromise the structural integrity of the device, 3) lead to the device not performing as intended, or 4) create a risk of contamination and cause the transmission of infectious diseases resulting in patient injury, illness, or death.

For product code 12601 only:

⚠ Caution: This medical device contains DEHP (diethylhexylphthalate) that is presently classified in the European Union as a presumed human reproductive toxicant based on data from animal studies. There is no conclusive scientific evidence that exposure to DEHP contained in medical devices has caused harmful effects in humans. A risk assessment, which took into account DEHP exposure of all indicated patient populations, including those who are potentially at increased risk, was performed for this device and the conclusion is that the device is safe when used as directed.

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*Registered Trademark or Trademark of Avanos Medical, Inc., or its affiliates. © 2018 AVNS. All rights reserved.
Distributed in the USA by Avanos Medical Sales, LLC, 5405 Windward Parkway, Alpharetta, GA 30004 USA.
Avanos Medical, Inc., 5405 Windward Parkway, Alpharetta, GA 30004 USA.
Sponsored in Australia by Avanos Medical Australia Pty Ltd, 475 Victoria Avenue, Chatswood, NSW 2067, Australia.