


BALLARD* ASSISTED CARE 24-HOUR ORAL CARE KIT WITH SUCTION TOOLS

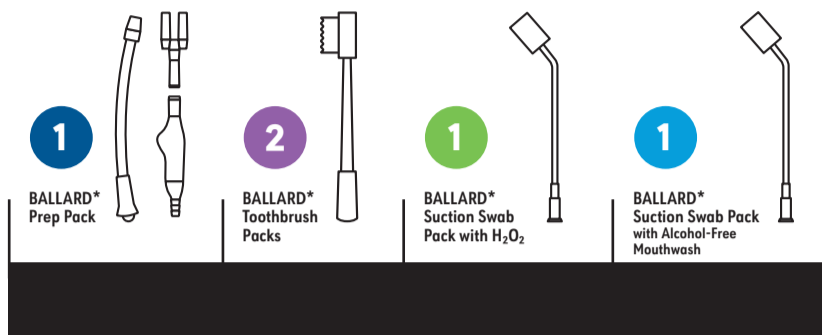

Instructions for Use: Oral care should be performed per institutional protocol.

** For single patient use. Patients with altered levels of consciousness or who cannot comprehend commands may require use of a bite block. See instructions for use below.

PREP PACK ASSEMBLY



- Select **BALLARD* Prep Pack** from kit and open package.
- Connect stepped adapter of suction handle firmly to suction canister tubing. Attach Y-adapter firmly to suction port. Then firmly attach oral care suction tubing to one Y-adapter port and closed suction tubing to the other port. (Use the Y-adapter only if using a single suction canister for both closed suctioning and oral care.)
- Attach Yankauer firmly to suction handle.
- To store between uses, clip suction handle to top of suction tubing in upright position.

YANKAUER ASSEMBLY AND USE

- Attach Yankauer firmly to suction handle.



- Slide cleaning sleeve toward suction handle.
- To apply suction, slide switch to ON.
- Suction and clean oral cavity.
- After suctioning, advance cleaning sleeve toward tip to clean and dry the Yankauer.




- With suction still on, immerse Yankauer tip in sterile water or appropriate solution to clear Yankauer and suction tubing.
- Slide switch to OFF. (Switch should remain in OFF position when Yankauer is not in use.)
- To store, clip suction handle to top of suction tubing in upright position.
- **Note:** discard tubing, Y-adapter, suction handle, and Yankauer after using for 24 hours or according to protocol.

TOOTHBRUSH PACK with Antiplaque Solution

SUCTION SWAB PACK with H₂O₂

SUCTION SWAB PACK with Alcohol-Free Mouthwash

ASSEMBLY AND USE

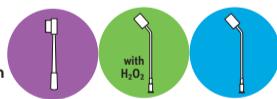

Toothbrush or Suction Swab

- Select **BALLARD* Toothbrush or Suction Swab Pack** from kit and open package.
- Remove Yankauer from suction handle and set in component tray. Firmly attach tool.
- Peel back solution dispenser lid and moisten tool†
- To apply suction, slide switch to ON.
- Clean and suction oral cavity using appropriate tool according to hospital protocol.
- Slide switch to OFF.
- Remove tool from suction handle and discard.
- Reattach Yankauer firmly to handle and clip to suction tubing.

Applicator Swab and Mouth Moisturizer

- Remove applicator swab and coat with moisturizer.
- Apply moisturizer to lips and mouth.
- Discard after use.

† Suction Swab Pack with H₂O₂ provided for 24-hour period per Drug Facts.

**See graphics on the inner panel of the kit for a recommended order of use of components to support delivery of oral care by cleaning, refreshing, and stimulating the tissues of the mouth and throat, and subsequently suctioning excess liquids remaining after cleaning.

Nonsterile	Disposable	Not made with natural rubber latex
For use by a healthcare professional	Store at room temperature	REF 970600