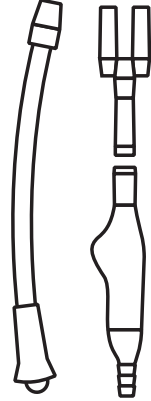
ΔVΔNOS*

BALLARD* ORAL CARE PREP PACK

Instructions for Use

- FR Pochette de préparation pour soins buccaux
- DE Mundpflege-Prep-Packung
- RU Комплект за подготовка на устната кухина
- ES Paquete preparatorio Oral Care
- CS Balíček pro přípravu ústní péče
- DA Klargøringspakke til oral behandling
- ET Suuhoolduse ettevalmistuskomplekt
- EL Πακέτο προετοιμασίας στοματικής φροντίδας
- IT Confezione di preparazione per igiene orale
- LV Mutes dobuma aprūpes sagatavošanas komplekts
- LT Burnos priežiūros sistemos pagrindinis procedūrinis komplektas
- HU Oral Care előkészítő csomag
- NL Preparatiepakket voor mondverzorging
- NO Munnpleie klargjøringspakke
- PL Pakiet przygotowawczy do pielęgnacji jamy ustnej
- PT Conjunto de preparação de higiene oral
- RO Pachet pregătitor pentru îngrijire orală
- RU Набор для предварительной обработки полости рта
- SK Balíček na prípravu ústnej starostlivosti
- SL Paket za pripravo za ustno nego
- FI Suunhoidon valmistelupakkaus
- SV Rengöringspaket för munvård
- TR Ağız Bakımı Hazırlık Paketi
- JA 口腔ケア準備パック
- KO 구강 케어 준비 팩
- ZH 口腔护理准备包

Instructions for Use

(E) NOTE: For use with BALLARD* 24-Hour Oral Care Kits, BALLARD* Packs and BALLARD* or READY CARE* Components. Oral care should be performed per institutional protocol. For single patient use. Use carefully for patients with altered levels of consciousness, may require use of a bite block.

 Single use only	 Not made with natural rubber latex	 Caution	 Consult instructions for use	For use by a healthcare professional
---	--	---	--	--------------------------------------

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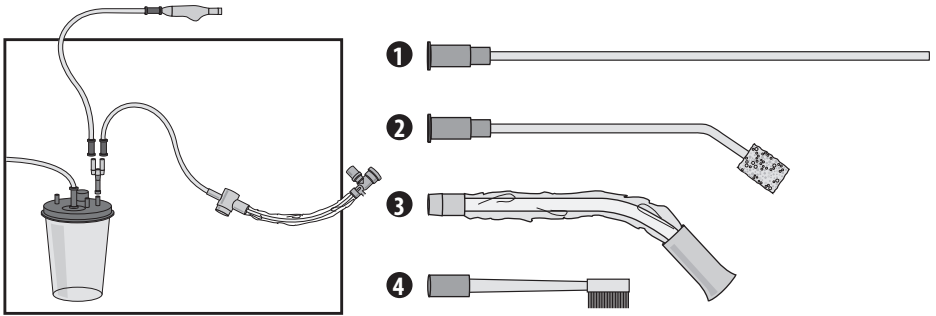
①

ⒺⓃ **Catheter**

NOTE: Catheter, Suction Swab, and Toothbrush sold separately.

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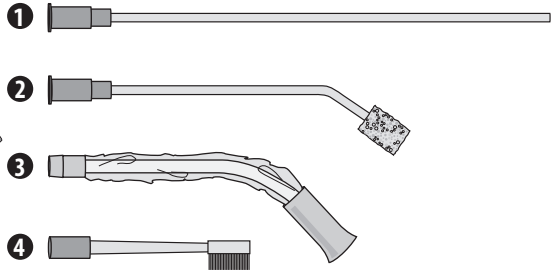
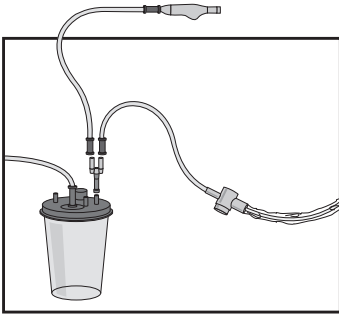
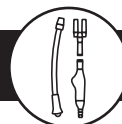
EN **BALLARD* Prep Pack**

Contains:

- 1 Self-Cleaning, Covered Yankauer
- 1 Suction Handle
- 1 Y-Adapter

BALLARD* Prep Pack Assembly and Use

- Select BALLARD* Prep Pack and open package.
- Connect stepped adapter of Suction Handle firmly to suction canister tubing. Attach Y-Adapter firmly to suction port. Then firmly attach oral care suction tubing to one Y-Adapter port and closed suction tubing to the other. (Use the Y-Adapter only if using a single suction canister for both closed suctioning and oral care.)
- Connect BALLARD* Oral Care Components as needed to smooth adapter of Suction Handle.
- After use, slide switch to OFF. Remove tool from Suction Handle and discard tool.
- To store between use, clip Suction Handle to top of suction tubing in upright position.

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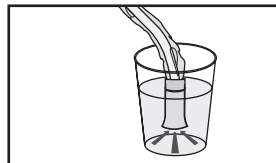
Figure 2

Yankauer Assembly and Use

- Attach Yankauer firmly to Suction Handle.
- Slide cleaning sleeve toward Suction Handle. **(Figure 1)**
- To apply suction, slide switch to ON.
- Suction and clean oral cavity.
- After suctioning, advance cleaning sleeve toward tip to clean and dry the Yankauer. **(Figure 2)**
- With suction still on, immerse Yankauer tip in sterile water or appropriate solution to clear Yankauer and suction tubing.
- Slide switch to OFF. (Switch should remain in OFF position when Yankauer is not in use.)
- To store, clip handle to top of suction tubing in upright position. Note: discard tubing, Y-Adapter, Suction Handle, and Yankauer after using for 24 hours or according to protocol.

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*Registered Trademark or Trademark of Avanos Medical, Inc., or its affiliates.
© 2018 AVNS. All rights reserved. 2019-08-29
Distributed in the USA by Avanos Medical Sales, LLC,
Avanos Medical, Inc., 5405 Windward Parkway, Alpharetta, GA 30004 USA.
Sponsored in Australia by Avanos Medical Australia Pty Ltd,