



# BALLARD\* mini 24-HOUR ORAL CARE KIT | Q4

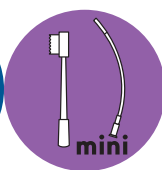


1

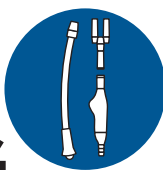

BALLARD\*  
Prep Pack

2

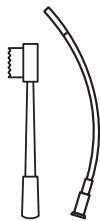

BALLARD\*  
Toothbrush and  
mini Suction Catheter (8 Fr) Packs


4




BALLARD\*  
mini Suction Swab Packs  
with Alcohol Free Mouthwash

**Instructions for Use:** Oral care should be performed per institutional protocol.\*\*  
For single patient use. Patients with altered levels of consciousness or who cannot comprehend commands may require use of a bite block. See instructions for use below.

## PREP PACK ASSEMBLY

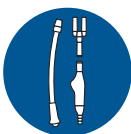

- Select BALLARD\* Prep Pack from kit and open package.
- Connect stepped adapter of handle to suction tubing. Attach Y-connector to suction port. Then attach oral care suction tube to one Y-connector port and closed suction tube to the other. (Use the Y-connector only if using a single suction canister for both closed suctioning and oral care.)
- Attach Yankauer to handle.
- To store, clip handle to top of suction tubing in upright position.

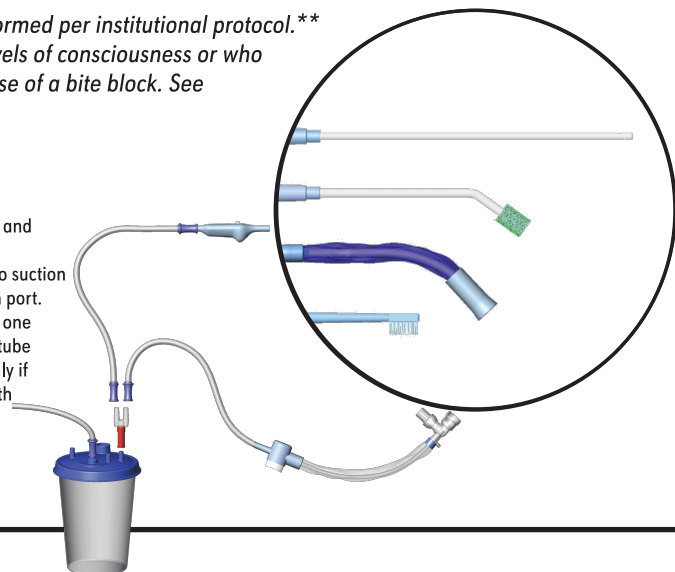


## YANKAUER ASSEMBLY AND USE

- Attach Yankauer to handle.
- Slide cleaning sleeve toward handle.
- To apply suction, slide switch to ON.
- Suction and clean oral cavity.
- After suctioning, advance cleaning sleeve toward tip to clean and dry the Yankauer.
- With suction still on, immerse Yankauer tip in sterile water or appropriate solution to clear Yankauer and suction tubing.
- Slide switch to OFF. (Switch should remain in OFF position when Yankauer is not in use.)
- To store, clip handle to top of suction tubing in upright position.
- **Note:** discard tubing, Y-adapter, handle, and Yankauer after using for 24 hours or according to protocol.



## TOOTHBRUSH AND mini SUCTION CATHETER PACK ASSEMBLY AND USE

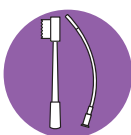

- Select BALLARD\* Toothbrush from kit and open package.
- Remove Yankauer from handle, set aside and attach Toothbrush.
- Peel back solution dispenser lid and moisten Toothbrush.
- Clean and suction oral cavity according to facility protocol.
- To apply suction, slide switch to ON.
- After suctioning is complete, slide switch to OFF.
- Remove Toothbrush from handle and discard. If needed, attach Suction Catheter to handle.
- To apply suction, slide switch to ON.
- Suction and clean oropharyngeal cavity.
- After suctioning is complete, slide switch to OFF.
- Remove Catheter from handle and discard.
- Reattach Yankauer to handle.

### Applicator Swab and Mouth Moisturizer in each pack.

- Remove applicator swab and coat with moisturizer.
- Apply moisturizer to lips and mouth.
- Discard after use.

## mini SUCTION SWAB PACK ASSEMBLY AND USE



- Select Suction Swab from kit and open package.
- Remove Yankauer from handle, set aside and attach Suction Swab.
- Peel back solution dispenser lid and moisten swab.
- Clean and suction oral cavity according to facility protocol.
- To apply suction, slide switch to ON.
- After suctioning is complete, Slide switch to OFF.
- Remove Suction Swab from handle and discard.
- Reattach Yankauer to handle.

### Applicator Swab and Mouth Moisturizer in each pack.

- Remove applicator swab and coat with moisturizer.
- Apply moisturizer to lips and mouth.
- Discard after use.

\*\*See graphics on the inner panel of the kit for a recommended order of use of components to support a typical q4h oral care protocol. The mini Suction Catheter Packs may be used as needed. For convenience, the start time may be entered in the blank arrow.

Nonsterile	Disposable	Not made with natural rubber latex	For use by a healthcare professional	Store at room temperature	REF <b>970008</b>
------------	------------	------------------------------------	--------------------------------------	---------------------------	-------------------