

### Suggested Cleaning Instructions for Hospital Use

Salter Labs® 8900 Series jet nebulizers are single-patient, multiple-use, and disposable. The nebulizers should be replaced every 7 days, or after 21 cleaning cycles, whichever is first. Replace the nebulizer sooner, if it becomes damaged, contaminated or is not functioning properly.

Cleaning procedures may vary depending on the patient population, disease process, and the hospital's infection control policies. Always follow your facility's policy and procedure for the delivery of aerosol treatments and the cleaning of equipment. AirLife® recommends the following cleaning procedures for the 8900 small volume nebulizer series.

### Cleaning Method

After each use, thoroughly rinse the nebulizer with sterilized water. DO NOT submerge the supply tubing in water.

1. Disassemble all nebulizer components; do not remove any of the one-way valves.
2. Rinse with sterilized water, shake out any excess.
3. Allow the device to dry.
4. Re-assemble and place in a dry, dust free location.
5. Replace the nebulizer every 7 days, or after 21 cleaning cycles, whichever is first.

Always follow your facility's policy and procedure for the delivery of aerosol treatments and the cleaning of equipment.