

User Guide

7100 BUBBLE HUMIDIFIER HOME USE



7100 High Flow Bubble Humidifier 3 PSI Safety Pop-Off- 2 LPM to 5 LPM

Please read and follow the Instruction for Use provided with the product before using the bubble humidifier

The 7100 Bubble Humidifier is a disposable, single patient use device for use in health care facilities and home care. The humidifier is used to provide additional humidity to oxygen therapy for flow rates at 2 to 5 LPM. The Bubble Humidifier has a 3 PSI safety pop-off valve, which sounds and releases excess pressure if the oxygen tubing becomes blocked.

Product Use

1. Wash hands and remove the Bubble Humidifier from packaging.
2. Use sterile, distilled or boiled tap water. If using boiled tap water, cool water to room temperature before adding to the Bubble Humidifier.
3. Unscrew the lid and add water to the maximum level marking (290 mL) located on the side of the bottle.
4. Replace the lid. Ensure that the threads are properly aligned and that the lid is screwed on tight.
5. Attach the Bubble Humidifier to an oxygen source, e.g. flowmeter, oxygen concentrator or liquid tank.
6. Test for leaks. Turn the flow to 3 LPM and block the oxygen outlet port on the Bubble Humidifier (A). A sound should emit within 5 seconds. If there is no sound, check the lid and other connections for leaks. Repeat the leak test.
7. Attach the supply tubing of the nasal cannula or mask to the humidifier's outlet port (B) and adjust oxygen flow to the prescribed setting.
8. Check for bubbles in the water. If there are no bubbles check for leaks. If the device sounds, check for tubing obstruction.
9. Check for flow from nasal cannula or mask and place device on face.
10. If water level drops below the minimum level marking on the Bubble Humidifier, add more water
11. Empty and replace the water in Bubble Humidifier daily.

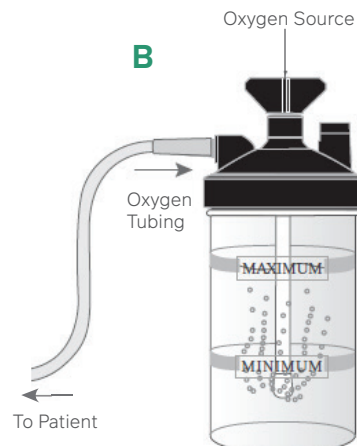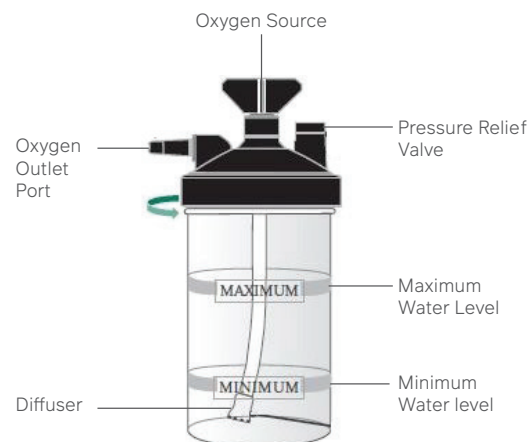

User Guide

7100 BUBBLE HUMIDIFIER HOME USE

7100 High Flow Bubble Humidifier 3 PSI Safety Pop-Off- 2 LPM to 5 LPM

Cleaning Instructions


1. Every 14 days wash the Bubble Humidifier with warm, soapy water. Use a non-scented, lotion free liquid dish detergent. If you use tap water in your Bubble Humidifier, clean the Bubble Humidifier every 3 days to help prevent buildup of mineral deposits. Do not use bleach or ammonia solutions to clean your device.
2. Rinse the Bubble Humidifier thoroughly with clean water.
3. After cleaning the device, visually inspect to ensure that it is clean. Repeat cleaning if necessary.
4. Replace the Bubble Humidifier every 90 days (3 months), or sooner if unable to remove mineral deposits by cleaning.

Note

Using flow rates outside the recommend range of 2 LPM to 5 LPM and a source pressure less than 5 PSI may affect the function of the safety pop-off valve.

WARNING

Leaks may partially or fully reduce the oxygen being delivered.

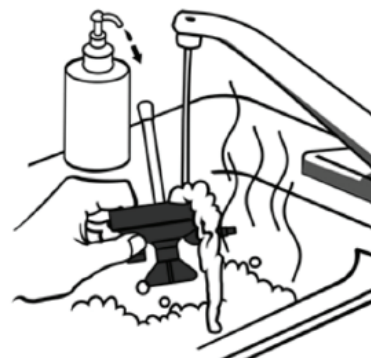

Your setup may vary slightly depending on what system you use for your oxygen source.